AT51002

INSTRUKCJA OBSŁUGI BALKONIK
USER'S MANUAL WALKER
GEBRAUCHSANWEISUNG ROLLATOR
NÁVOD K POUŽITÍ CHODÍTKA
NÁVOD NA POUŽITIE CHODÍTKO
MANUEL D'INSTRUCTION DU DÉAMBULATEUR
HANDLEIDING LOOPREK
INSTRUCCIONES DE USO DEL ANDADOR
ISTRUZIONI PER L'USO DEL DEAMBULATORE
INSTRUKTIONER FÖR ANVÄNDNING AV ROLLATORN

IU_AT51002

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DESTINATION

The folding walker is intended for people with problems with standing up, with musculoskeletal disorders or with balance problems when walking.

PRODUCT FEATURES

The walker provides the user with support, reducing the risk of falling over. The product is suitable for learning to walk or following an injury.

- made of reinforced aluminium
- equipped with a set of two castors and feet
- 3 in 1; can be used as a walker with 4 feet, a staggered walker or a twin-wheeled walker to facilitate user's mobility
- Easy to fold and transport, foldable at one touch of a button
- 8 steps of height adjustment
- ergonomic handles
- easy to store thanks to the foldable frame.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling.

WARNINGS:

1. Before using the walker, please read these operating instructions and keep them for future reference.
2. Use of the walker on unsupportive surfaces such as wet, icy, greasy or very dusty ground is not permitted because of the risk of falling.
3. Before each use, - perform a visual inspection of all structural components of the product including: - carefully check the stability and condition of the supporting frame.
4. The walker is for one user only, it is forbidden to stand on the walker.
5. The maximum height level should not be exceeded.
6. There is a possibility of jamming of fingers in the folding area of the frame of the walker.
7. Do not use the walker if there is any visible or perceptible damage (e.g. breakage of the frame) as this may cause an accident.
8. Do not lean outside the perimeter of the walker. This may cause the user to fall over.
9. Do not rock the walker. This may cause the user to fall over.
10. Do not use if the weight of the user exceeds 115 kg.

11. Avoid contact with open flame. The product may ignite.

12. Ensure that the foot or castor retaining clip pin is at a uniform height after installation. Otherwise, the product may not be stable and the hole may deform.

ATTENTION:

In the event of a product-related „serious incident“ that directly or indirectly led to, could have led to or is likely to lead to any of the following events:

- a) the death of a patient, user or other person; or
- b) temporary or permanent deterioration of the health of a patient, user or other person, or
- c) a serious threat to public health

This „serious incident“ should be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident.

ATTENTION:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult your healthcare professional.

TECHNICAL SPECIFICATIONS:

Height: 75-93 cm, 8 steps adjustable in 2 cm increments

Width between handles: 46 cm

Weight with castors: 2,650 kg, with feet: 2.35kg

Depth: 40 cm

Thickness when folded: 12 cm

Wheel diameter: 12,5 cm

Width: (front) 46 cm, (back) 51 cm

Maximum user weight: 115 kg

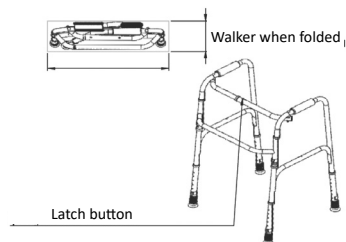
**THIS SIGN INDICATES THE
MAXIMUM WEIGHT OF THE USER.**

MAINTENANCE OF THE PRODUCT

The product should be cleaned periodically with water and cleaning agents. The product can be treated with disinfectants. When not in use, the product should be stored in a dry area. Avoid prolonged exposure to direct sunlight.

ATTENTION: The manufacturer shall not be liable for damage caused by negligent maintenance, inadequate servicing or as a result of failure to observe the instructions in this manual.

PREPARATION FOR USE:

To fold the walker, press the latch button and pull the sides of the walker inwards, then fold the walker.

WALKING WITH THE "STEPPING" WALKER

The walking function is activated when the stepping function locking pins are in the front reinforcement bar. Stand by the walker holding the handles of the walker with both hands. This type of equipment is moved by alternately lifting the sides of the walker upwards and moving them forwards, and by placing the lower limbs in front.

WALKING WITH THE WALKER WITH THE FIXED FUNCTION ACTIVATED

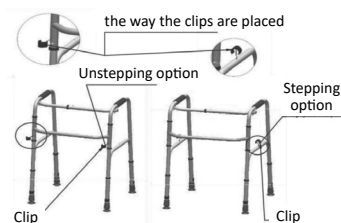
To obtain the fixed function of the walker, remove the two safety pins located on the upper horizontal part of the frame reinforcement bar and put them into the holes located at the same height on the vertical part of the frame. Moving around with the fixed walker is done by moving it forwards and then moving the limbs. If the walking function is to be restored or if the walker is to be folded, the pins must be re-inserted in the horizontal reinforcement bar.

WALKING WITH A WHEELED WALKER

To obtain a walker with wheels, first lock the walker (switch off the stepping function), remove the front legs and fit the wheels in their place. Remember to

check the height of all the legs (they must be the same length).

Before using the product, the height must be adjusted. To do this, push in the clip and lock it at the desired level - the same for all four legs. Otherwise, the product may not be stable and the hole may become deformed.

TARGET PATIENT GROUP

The healthcare professional shall indicate the use of the device to adults and children at his/her own responsibility, taking into account the available variants/sizes/necessary functions and indications, bearing in mind the information provided by the manufacturer.

STORAGE

The product should be stored in a dry cool place, away from moisture and direct sunlight.

WASTE TREATMENT

After decommissioning, the product may be treated as municipal waste. Segregation at end-of-life is recommended.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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