

WARRANTY CARD

Model: .....

Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

- 1) ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
- 2) During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre together with the proof of purchase and the warranty card.
- 3) ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
- 4) If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
- 5) Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
- 6) The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
- 7) THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
- 8) Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.



ANTAR Sp. J.  
ul. Zawiślańska 43, 03-068 Warsaw,  
Poland;  
Phone no. +48 22 518 36 00;  
E-mail: antar@antar.net;  
www.antar.net



**User's manual**  
**Shower toilet stool**  
**Models:**  
**without backrest (AT51009),**  
**with backrest (AT51010)**



Instruction for use version: 2021-05-25.1

Date of issue of the instruction for use: 2021-05-25

### Shower toilet stool

**Models: without backrest (AT51009),  
with backrest (AT51010)**

Shower toilet stools are to be used by mobility impaired persons who find it hard to take a shower.

A shower toilet stool is provided with a height adjustment mechanism in order to provide users with as much comfort as possible.

The product is provided with anti-slippery protections. Once the user stops using the stool, they should get it clean by washing it thoroughly.

Regular detergents and disinfection products can be used.

Once worn out, the product should be handled as requested for municipal waste, unused should be storage in dry room.

Composition of the product: aluminum frame, seat and backrest - PE.

### AT51010



### AT51009



Maximum user weight:	120 kg
Minimum height:	39 cm
Maximum height:	49 cm
Seat width:	50 cm
Product weight:	3 kg

Maximum user weight:	120 kg
Product weight:	2,2 kg
Seat width:	50 cm
Total width:	50 cm
Seat depth:	29 cm
Seat thickness:	2,5 cm
Minimum height:	35 cm
Maximum height:	45 cm



This symbol means: "Maximum user weight"

#### WARNING:

While using a stool with a support (optional) proceed with utmost care. You should check whether the product was correctly assembled. You must not lean out too much, particularly backwards or otherwise you can fall down.

#### WARNING:

In case a serious incident that has occurred in relation to the medical device, that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat;

the above mentioned serious incident should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established;

#### WARNING:

In case any pain, allergic reaction or other worrying symptoms, unclear for the user which have occurred in relation to the medical device, the user should consult a healthcare professional;

#### SAFE DISPOSAL OF THE MEDICAL DEVICE

When the device has been withdrawn from use it can be treated as municipal waste except for electric medical devices – in such case the electric medical device shall be disposed in the way described by the law for electrical and electronic waste disposal.