USER MANUAL - ROLLATOR AT51111
GEBRAUCHSANWEISUNG - AT51111 ROLLATOR
INSTRUCCIONES DE USO - AT51111 andador
MODE D'EMPLOI - AT51111 déambulateur
ISTRUZIONI PER L'USO - AT51111 deambulatore
GEBRUIKSAANWIJZING - AT51111 andador
ANVÄNDARMANUAL - ROLLATOR AT51111

WARNING:

In the event of a device-related "serious incident", which directly or indirectly led to, could have led to, or is likely to lead to any of the following events:

- a) death of a patient, user or another person, or
- b) temporary or permanent deterioration of the health of a patient, user or another person, or
- c) a serious risk to public health, such a "serious incident" must be notified to the manufacturer and to the competent authority of the Member State, in which the user or patient resides. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNING:

In the event of pain, allergic reactions or other alarming, unclear symptoms related to the use of the medical device, please consult a healthcare professional.

METHOD OF DISPOSAL OF THE PRODUCT AFTER REMOVING FROM SERVICE

After the device is withdrawn from use, the medical device can be disposed of as normal household waste, except for electrical products - proceed in the manner appropriate for the disposal of electrical and electronic equipment.

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

Characteristic

The AT51111 Rollator Rollator is designed for indoor and outdoor use. The device has been designed to be used only as a walker with a seat for one person. It is forbidden to transport more than one person. Unless otherwise stated, these instructions for use are intended for the patient as the user of the walker.

Destination

A rolling walker is intended for people with limited mobility who still have the ability, strength, and balance to leave the house with the help of a wheeled walker.

Contraindications

Cases in which the use of a walker is not recommended:

- Problems with perception
- Serious problems with balance
- Hands are not allowed
- Inability to sit

ATTENTION:

When using and handling the walker, when folding it and adjusting the mechanisms, there may be a risk of entrapment and/or pinching of parts of the user's/companion's body in the openings/gaps between the elements.

These steps must be performed with extreme caution.

After completing the adjustment, stabilize the position by carefully tightening the nuts/bolts.

The expected useful life of a rolling walker is a maximum of 5 years, provided that the rules of operation, safety, maintenance and service are followed. A walker that is still in good condition can still be used after this period.

Responsibility

As the manufacturer, we are only responsible for the safety, reliability, and suitability of the rolling walker, provided that modifications, upgrades, repairs, and maintenance are performed by authorized persons and the rollator is used in accordance with the instructions contained herein.

Facility**Handle height**

The height of the handle can be adjusted to your own height. This operation does not require the use of any tools.

There is a red padlock under each handle (Fig. 1). The lock must be pressed and held. In this configuration, you can lower or raise the handle. Release the lock. Pull or push the lock again until you hear a click, which means the lock is securely in place and cannot be moved any further. Repeat the above steps on the other side so that both handles are at the same height.

Fig. 1

Brakes

Have a trained professional adjust the brakes. However, if you have the right experience, you can do it yourself. However, it must be remembered that the incorrect installation of the brakes can be life-threatening.

Increase/decrease braking force

1. Loosen the socket head screw
2. Tighten or loosen the Bowden cable by pulling or forcing it
3. Tighten the hex head screw.
4. Fine adjustment:

Loosen the locknut until you can turn the adjusting screw.

Turn the adjusting screw clockwise: reduce the braking force. Turn the adjusting screw counterclockwise: increase the braking force.

Finally, tighten the locknut again.

Repeat the procedure for the second brake.

The brakes should be adjusted so that they do not create resistance when moving, but allow easy movement. Locked brakes keep the walker from moving.

Opening and folding of the walker

To save space during storage, the rollator can be folded.

To use it again, open the rollator by pressing on the outer edges of the seat (fig. 2). There is a block at the bottom of the walker that prevents it from accidentally folding. An audible click when unfolding the walker means the lock is working properly. To fold the walker, pull up on the seat handle (fig. 3).

Fig. 2

Fig. 3

Front wheels

To release the front caster lock, press the red latch. Then pull the front wheels forward and turn them 180° up. To use the walker again, pull the front wheels forward and turn them down.

One click means the wheels are in the correct position; the seat can now be adjusted accordingly.

Rear wheels

Then fold the lever. In the last step, press the button and slide the rear wheel mechanism down. Please note that the rear wheels are always connected to the walker via the brake cable.

To unfold them, push the rear wheel mechanism back on the frame. One click means the wheels are in the correct position. The lever should now fold back.

Make sure the rear wheels are securely fastened and the brake is working properly.

Footrests to facilitate tilting

The rollator is equipped with two tilting footrests, designed to make it easier to get over curbs. By stepping on the footrest with your foot and at the same time pulling the handles with your hands, the front wheels are raised. This makes it easier to walk over curbs and get over small obstacles.

When performing such a maneuver, remember to keep your balance.

Brake and handbrake

The rollator is equipped with a manual brake and a parking brake to prevent accidental movement of the rollator.

To stop the walker easily and smoothly, lift both brake levers.

Braking performance depends on how hard the brake levers are pressed. To engage the parking brake, push the brake levers down; It means that the lever is locked in the correct position.

Pull the brake levers up to release the parking brake.

Use a rolling walker as a seat

The rolling walker has a seat with a backrest. Before sitting on the seat, make sure the parking brake is on. Otherwise, the walker could move away and cause you to fall. Additional pillows cannot be used; they can slip and cause you to fall.

Cleaning

Frame parts can be cleaned with a damp cloth. A mild cleaning agent can be used for stubborn dirt.

The wheels can be cleaned with a damp brush with plastic bristles (not wire brushes!).

The backrest, seat and storage compartment can be washed with mild soap and water.

Disinfection

The walker can be disinfected with common household disinfectants. Approved alcohol-based disinfectant sprays or wipes should be used for this purpose.

After disinfection, allow the rollator to air dry, adhering to the operating times specified by the disinfectant manufacturers.

RE-USE

The AT51111 rollator is reusable. The rollator must be cleaned and disinfected in advance according to the attached information on maintaining its hygiene. When passing the rollator to another person, remember to also provide the new user with all the necessary technical documents. The walker must be checked by an authorized specialist and transported in the correct conditions.

ELIMINATION

The rollator AT51111, which will no longer be used, must be discarded; contact your dealer for this. If you wish to dispose of it yourself, contact your local waste disposal company for information on how to dispose of it.

Dimensions and weight	
Seat width	450 mm
Seat height	540 mm
Overall width	653 mm
Overall length	706 mm
Total height /	
The height of the handle (adjustable in 6 positions)	810 - 936 mm
Curb weight (without accessories)	7.85 kg
Load	136 kg
The maximum load of the pannier is	5 kg
Folded dimensions:	706 x 250 x 810 mm (L x W x H)
Wheels (with protection)	
Swivel wheels (front)	250 x 35 mm
Brake wheels, non-swivel (rear)	210 x 35 mm
Turning radius	1770 mm
Materials	
Aluminum frame (anticorrosive)	
Black nylon seat	
Backrest Nylon, black	
Polyurethane tires	
Operating conditions	
Temperature	0-50°C
Humidity	20 - 80%
Storage conditions	
Temperature	-20 to + 60°C
Humidity	45 - 60%

Delivery range

The delivery includes the following main components:

- 1 outer package
- Rollator walker with rear strap and tilting footrests
- mesh bag with reflector and zipper
- 2 horizontal padded handles
- instructions for use
- 1 position LED lamp

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

- 1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE).
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warszawa, ul. Zawisłańska 43
Polska
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net e-mail:antar@antar.net

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