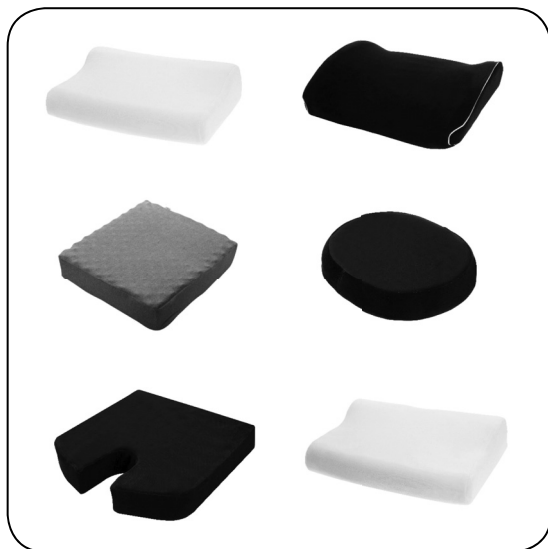
**AT03001, AT03002, AT03003, AT03004, AT03005,
AT03006, AT03006 D, AT03007, AT03008, AT03009, AT03010,
AT03011, AT03012, AT03014, AT03015, AT03016, AT03017**

**PODUSZKA ORTOPEDYCZNA INSTRUKCJA OBSŁUGI
ORTHOPAEDIC PILLOW INSTRUCTION MANUAL
ORTHOPÄDISCHES KISSEN BEDIENUNGSANLEITUNG
NÁVOD K POUŽITÍ ORTOPEDECKÉHO POLŠTÁŘE
NÁVOD NA POUŽITIE ORTOPEDECKÉHO VANKÚŠA
MANUEL D'INSTRUCTION POUR OREILLERS ORTHOPÉDIQUES
HANDLEIDING ORTHOPEDISCH KUSSEN
MANUAL DE INSTRUCCIONES DE LA ALMOHADA ORTOPÉDICA
MANUALE D'ISTRUZIONI DEL CUSCINO ORTOPEDICO
ORTOPEDISK KUDDE BRUKSANVISNING**

IU_AT03001 - AT03017

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GENERAL WARNING

Do not use the product before reading and understanding these instructions. If you do not understand the warnings, cautions or instructions, contact your healthcare professional or dealer to avoid damage to the product.

PURPOSE/INDICATIONS

Depending on the type of pillow, they have different uses. The sleeping pillow is primarily intended for people who suffer, for example, from tension and migraine headaches and pain and strain syndromes of the spine.

The task of the lumbar pillow is to maintain the natural position of the spine, supporting it while maintaining a prolonged sitting position. Seat cushions prevent pains caused by prolonged sitting, especially in wheelchair users.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

DIMENSIONS

AT03001: length 44 cm, width 30 cm, height 7 to 11 cm

AT03002: length 44 cm, width 30 cm, height 7 to 11 cm

AT03003: length 42 cm, width 35 cm, height 7 to 10 cm

AT03004: Width 33 cm, Height 22 cm, Thickness 7 cm

AT03005: width 33 cm, length 33 cm, height 11 cm

AT03006/AT03006 D (in disinfection cover):

- S: 42 x 38 x 8 cm,

- M: 43 x 43 x 8 cm,

- L: 46 x 43 x 8 cm,

- XL: 48 x 43 x 8 cm

AT03007: length: 47 cm, width: 32 cm, height: 7 - 12 cm

AT03008: 45 x 40 x 10 cm, 45 x 40 x 6 cm

AT03009: 41 x 32 x 8 cm

AT03010: 45 x 40 x 10 cm, 45 x 40 x 6 cm

AT03011:

- S: 42 x 38 x 8 cm,

- M: 43 x 43 x 8 cm,

- L: 46 x 43 x 8 cm,

- XL: 48 x 43 x 8 cm

AT03012: 34 x 40 x 8 cm

AT03013: 40 x 40 x 8 cm, - 37 x 37 x 8 cm

AT03014: length: 55 cm, width: 31 cm height: 7 to 11 cm

AT03015: length: 60 cm, width: 31 cm height: 7 to 11 cm

AT03016: 660 x 440 x 111 mm

AT03017: 420 x 330 x 111 mm

CLEANING AND MAINTENANCE

Instructions for washing the cover (models: AT03001, AT03002, AT03003, AT03004, AT03005, AT03006, AT03007, AT03008, AT03009, AT03010, AT03011, AT03012, AT03014, AT03015, AT03016, AT03017)

- 1) Do not wash the foam!
- 2) Hand wash only the cover.
- 3) Clean with a damp sponge and detergent.
- 4) Maximum wash temperature 40°C.
- 5) Do not chlorinate! (Do not bleach the cover).
- 6) Do not dry clean!
- 7) Do not tumble dry! Dry laid out flat in the shade.
- 8) Do not iron! (avoid contact with heated surfaces e.g. radiator)
- 9) Deterioration of visual-aesthetic properties (such as surface wrinkling) does not alter the performance of the material.

Washing instructions for the disinfecting cover (AT03006D):

- 1) Do not wash the foam!
- 2) Clean using a damp sponge with detergent.
- 3) Maximum wash temperature 95 °C.
- 4) Do not chlorinate! (Do not bleach the cover).
- 5) Do not dry clean!
- 6) Do not tumble dry! Dry laid out flat in the shade.
- 7) Do not iron! (avoid contact with heated surfaces e.g. radiator).
- 8) Deterioration of visual-aesthetic properties (such as surface wrinkling) does not alter the performance of the material.

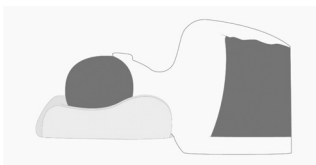
STORAGE

The product should be stored in a dry cool room, away from moisture and direct sunlight.

COMMENTS AND WARNINGS:

1. It is forbidden to use the product in children under 3 years of age. The soft structure of the sponge can seriously interfere with the breathing process when the baby is lying on his stomach.

2. Do not lie on the sleeping pillow in the prone position.
3. Avoid contact of the product with fire. The product may ignite.
4. The lumbar pillow should be placed on the backrest so that its flat part is in contact with the backrest, set at the height of the lumbar region.
5. The sleeping pillow (models: AT03001, AT03002, AT03007, AT03014, AT03015) should be placed so that the upper part of the pillow is under the neck, as shown in the picture below:

**CAUTION:**

In the event of a product-related „serious incident“ that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of the patient, user or other person or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above ‚serious incident‘ must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use

ATTENTION: The manufacturer is not responsible for damage caused by negligent cleaning, maintenance or as a result of not following the instructions in this manual.

TARGET PATIENT GROUP

The healthcare professional shall, under his own responsibility, indicate the use of the device to adults and children, taking into account the available variants/sizes/unnecessary functions/sizes and indications, bearing in mind the information supplied by the manufacturer.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

Once the device has been withdrawn from service, the medical device can be disposed of as normal municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to AN TAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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