USER'S MANUAL

Toilet lift seat AT51201, AT51202, AT51203, AT51204, AT51205

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Toilettensitzerhöhung AT51201, AT51202, AT51203, AT51204, AT51205

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Siège élévateur WC AT51201, AT51202, AT51203, AT51204, AT51205

GEBUIKERSHANDLEIDING

Toiletliftzitting AT51201, AT51202, AT51203, AT51204, AT51205

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Sedile per WC AT51201, AT51202, AT51203, AT51204, AT51205

GEBUIKERSHANDLEIDING

Toiletliftzitting AT51201, AT51202, AT51203, AT51204, AT51205

AT51201, AT51202, AT51203, AT51204, AT51205

If defects or damage occur, inform the supplier immediately. Do not use the product if you think it is damaged or missing parts.

Toilet lift seats are for people with arthritis, after hip surgery, and others who have difficulty moving, standing, and sitting. The booster seat has a modern restraint system that allows easy stabilization in the toilet. The ergonomic surface ensures an even distribution of compressive forces over the user's body. A slight inclined tilt of the seat forces the correct posture and reduces push-ups in the hip and knee joints. The reinforcement of the toilet has two incisions at the front and back, which facilitates hygienic activities in a sitting position.

Product features:
AT51201, AT51202, AT51203

- two mounting flaps adapt to the toilet bowl
- the installation of the booster seat is easy, you just need to screw two hooks
- the product is available in 2 heights: 10 and 15 cm (AT51201, AT51202)

AT51204:

- a secure attachment to the toilet with tightened clamps (more stable than with clippers) the mounting does not require any tools
- equipped with hinged handles to help a disabled person getting up from the toilet
- the seat riser fits to the most toilet bowls, it also has 3 levels of height adjustment

AT51205:

- secure attachment to the toilet
- with folding armrests to help a disabled person get up
- the booster seat fits most toilet bowls

Functionality:

- large openings at the front and back facilitate intimate hygiene
- can open the cover without touching the booster seat
- Can take a sitting position with a low lid

Technical data AT51201 , AT51202:

Size (width x length): 40 x 36 cm
 Maximum user weight: 180 kg
 Product weight: AT51201-1.2 kg, AT51202 -1.4 kg

Technical data AT51203

Size (width x length): 40 x 36 cm
 Maximum user weight: 136 kg
 Product weight: 2.2 kg
 Height: 10 cm
 Material: 100% PU (polyurethane)

Technical data AT51204:

Total height (cm): 37-59 (with raised cover) cm
 Size, width x length (cm): 40 (seat riser)/57 (with armrests) x 46
 Adjustable seat height (cm): 6, 10, 14
 Width between the upper parts of the armrests (cm): 47
 Product weight: 4.2 kg
 The user's weight and the maximum load must not exceed 136 kg

Technical data AT51205:

General depth. (cm): 35-60 (with raised bowl) cm x 46
 Size, width x length (cm): 40 (booster seat)/54 (with armrests)
 Width between the top of the armrests (cm): 47
 Product weight: 3.6 kg
 The weight of the user and the maximum weight of the load must not exceed 136 kg

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

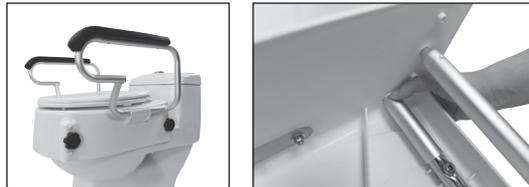
NOTE: It is forbidden to use the product contrary to its intended use!

Assembly AT51201 , AT51202, AT51203:


Connect the front member of the booster seat to the front of the toilet bowl, then screw the two fasteners with the side screws. The ergonomic and robust construction of the screws allows a precise and permanent installation of the booster seat. The fortifications were secured against slippage. The raised and lower covers are secured against accidental disassembly.

AT51201 , AT51202**AT51203****Assembly AT51204:**

- remove the two front plastic plugs
- loosen the screw and place the crossbar at the required height
- tighten it using two screws
- fix the rear crossbar in a similar way, both should be at the same height
- remove the existing toilet bowl, insert two threaded rods in the place of its fixing on the toilet bowl
- tighten the seat riser using plastic caps
- before first usage, make sure that the seat is correctly and securely mounted.

**Assembly AT51205:**

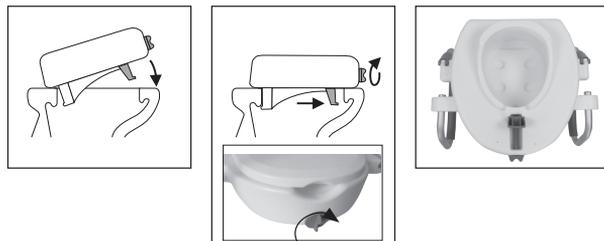- armrests should be attached to the toilet lift with screws located in the assembly with the armrests (requires allen 4 wrench)
- Connect the front member of the booster seat to the front of the toilet bowl, then firmly screw the accessory to the front of the booster seat using the screw. The ergonomic and robust construction of the screw allows a precise and permanent installation of the booster seat. The handle was secure against slipping.
- The raised and lower covers are secured against accidental disassembly.
- before the first use, make sure that the seat has been properly and safely mounted.

Life:

The expected service life of the toilet lift seat can be up to 5 years if used as intended, in accordance with safety and maintenance instructions. Beyond this period, the toilet seat lift will continue to be used in safe condition.

Cleaning:

- easy to clean, free of external rough surfaces
- do not use abrasive or scouring cleaning agents.
- if any defects or damage appear, please inform the supplier immediately. Do not use the product if it is suspected of being damaged or missing parts.
- clean it with hot water and soap or with soft detergents.

Disinfection by wiping

With commercially available disinfectants according to EN 12720

1. Put on protective gloves before preparing the product
2. Spray the areas to be treated with a disinfectant
3. Rub the treated areas with a paper towel, if necessary. Try again
4. Discard protective gloves and paper towels

Contraindications

- Troubled perceptual
- Severe balance disorders
- Inability to sit

METHOD OF DISPOSAL OF THE PRODUCT AFTER REMOVING FROM SERVICE

After the device is withdrawn from use, the medical device can be disposed of as normal household waste, except for electrical products - proceed in the manner appropriate for the disposal of electrical and electronic equipment.

NOTES

1. During use and during the assembly and adjustment of the mechanisms, there may be a risk that parts of the user/caregiver's body will be trapped and/or compressed in the openings/spaces between the elements.
2. After adjustment, stabilize the position by tightening the nuts/bolts.
3. Before each use of the product, check that all adjustable parts are securely attached.
4. Beware of wet or slippery surfaces.
5. Do not place your fingers in seat holes that drain water. This can cause your fingers to be compressed or pinched.
6. Do not use a defective or broken booster seat.
7. Keep the product away from children. The maximum load does not apply to the cover.
8. The blanket is not intended to sit.
9. The elevation of the toilet is not intended to be seated permanently.
10. By lowering the armrests, clothes or body parts can be pinched.
11. Missing, damaged or worn parts can only be replaced by an approved service.
12. Do not use pressure washers or harsh, corrosive detergents for cleaning.

WARNING:

In the event of a device-related "serious incident", which directly or indirectly led to, could have led to, or is likely to lead to any of the following events:

- a) death of a patient, user or another person, or
- b) temporary or permanent deterioration of the health of a patient, user or another person, or
- c) a serious risk to public health, such a "serious incident" must be notified to the manufacturer and to the competent authority of the Member State, in which the user or patient resides. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNING:

In the event of pain, allergic reactions or other alarming, unclear symptoms related to the use of the medical device, please consult a healthcare professional.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warszawa, ul. Zawiańska 43
Polska
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net e-mail: antar@antar.net

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