AT51040
INSTRUKCJA OBSŁUGI CHODZIK
USER MANUAL WALKER
GEBRAUCHSANWEISUNG ROLLATOR
MANUEL D'INSTRUCTION DU DÉAMBULATEUR
HANDLEIDING ROLLATOR

MD

PL

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ENG

INTRODUCTION

The rollators are manufactured from high-quality materials and include a tray, bag, and locking brakes. The hand-bar are adjustable in height and the rollator folds easily for transport or storage.

USE/PURPOSE

Designed to assist the user to walk in door, rest and carry small items of shopping for both indoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown below. The intended user should possess the necessary physical, visual, and cognitive skills to operate the rollator safely. If you have any doubt, please consult your dealer/prescriber. It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling.

SAFETY ADVICE

This rollator is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight between the handlebars with the unit in front of the user. Read this manual carefully before using your rollator.

USING

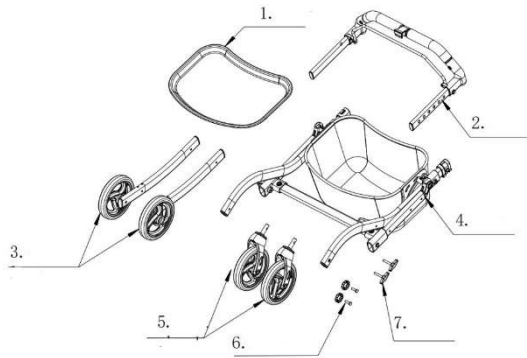
1. Always ensure the brakes are in the locked position if the rollator is left unattended
2. Check all castors and wheels are secure before use
3. Check the brakes are functioning properly
4. Check handle height adjustment hand knobs are secure before use
5. Check the rollator is fully unfolded
6. Take care when carrying the rollator when folded.
7. Keep your hands away from the folding mechanism to prevent any risk of pinching.

PRECAUTIONS

1. Always walking with the rollator in a smooth, controlled motion. Seek assistance if necessary.
2. Do not hang objects on the brake levers or handgrips. This could affect brake operation and/or stability of the rollator.

LIST OF COMPONENTS

1. Tray 2. handlebar 3. two rear legs 4. Main frame 5. front wheels 6. two Front screws 7. two "T" shape screws

SPECIFICATION

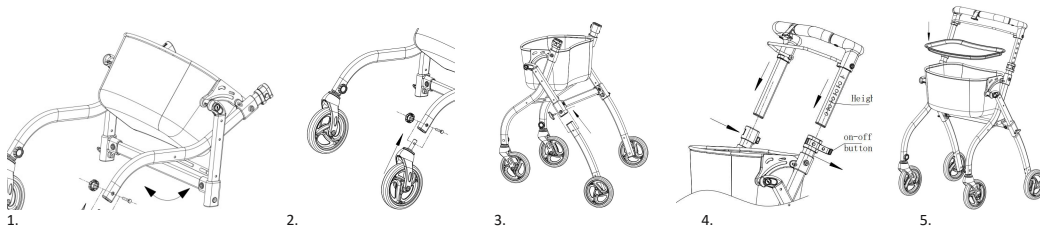
Overall width: 60cm
Overall length: 58cm
Height of tray: 67cm
Tray size: 39.5*27.5CM
Adjustable handle height: 85-97cm
Weight: 6.2kg
Pack size: 51*17*63cm
Solid wheels: 6"
Max user weight: 108kg

This symbol indicates the maximum weight of the user!

ASSEMBLY INSTRUCTION

1. Take out all packages.
2. Take out the main frame and open it as shown by the arrow. (1)
3. Insert front wheels into the main frame. (2)
4. Insert rear legs into the main frame (3)
5. First, Open the two on-off button (like picture) Secondly, Insert handle bar into the main frame. lock the on-off button (4)
6. Put the Tray on the main frame (5)

DEMONSTRATION

Perform all the above steps in reverse order.

ADJUSTABLE HANDLE

The handles of the walker can be adjusted in height to suit different users. Simply open the two switches and set them to the desired height. (4) Please make sure that the two sides are at the same height.

BRAKE OPERATION ADJUSTMENT

Pull up on the brake handle lever to brake. Push down on the brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock.

BRAKE ADJUSTMENT IS CARRIED OUT AS FOLLOWS:

Turn the adjusting screw anti-clockwise to increase the tension then lock the adjustment locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.
locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.

TIPS:

When using the indoor rollator, please adjust the height of the handrail to your favorite position so that you can sit on the chair in a more comfortable posture.

MAINTENANCE

Over a period of time, the brake cables may stretch, and if no further adjustments can be made the cables should be replaced. Seek the help of your dealer when replacing brake cables. The castors should be checked that they are secure each time before using the rollator. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the rollator appear loose or insecure discontinue use until it is inspected by an authorized dealer.

MAINTENANCE SCHEDULE:

Weekly

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs

Periodically

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tired for wear or cracks

Yearly

- Once a year take your rollator to an authorized repair perform a full check, clean, service and lubrication of all moving parts.

CLEANING

Your walker can be cleaned with a damp cloth or sponge and warm, soapy water. If necessary, gentle household antibacterial surface cleaners can be used. Take care not to leave any cleaning residue on the hand grips or seat (if fitted), which can make them slippery when in use. Do not use scrapers or rough brushes as this may damage surface coatings.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

TARGET PATIENT GROUP

The healthcare professional shall indicate the use of the device to adults and children on his/her own responsibility, taking into account the available variants/sizes/necessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

WARNINGS!

1. do NOT stand on this product.
2. make sure both push handles are set at the same height
3. always keep a safe distance between your walker and pedestrians or objects.
4. some components can cause damage or destruction if they come into contact with other objects.
5. do not attempt to replace the wheels or rollers of your walker. If they are faulty or need replacing contact your dealer.
6. make sure your walker is fully extended (folding scissor arm is straight) before using it again.

NOTES:

In the event of a device-related "serious incident" that directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In the event of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

CAUTION: During use and handling of the device, and when assembling and adjusting the mechanisms, there may be a danger of the user's/companion's body parts becoming trapped and/or compressed in the openings/gaps between components. Perform these operations with particular care. Once adjustments have been completed, stabilise the position by tightening the nuts/bolts firmly.

ATTENTION: It is forbidden to use the product other than for its intended purpose.

ATTENTION: Before use, check that all components have been assembled correctly.

ATTENTION: The manufacturer is not liable for damage caused by negligent maintenance, inadequate servicing or as a result of failing to observe the recommendations contained in these instructions for use.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

After the device has been taken out of service, the medical device can be disposed of as normal household waste.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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Antar Medizin GmbH