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Model:

Numer Serii:

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warszawa, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net
e-mail: antar@antar.net

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Admission

Thank you for purchasing our product. Before using, please read this user manual carefully. Incorrect use may lead to health detriment, injury or accidents. Therefore, in order to use the walker safely and successfully, please read this manual carefully.

NOTE: Inspect all parts for damage caused in transit. If any such damage is found, DO NOT use the product. More information from the manufacturer.

GENERAL WARNINGS

Do NOT assemble and use the product until you have read and understood this manual. If you do not understand any warnings, cautions, or instructions, consult your healthcare professional, dealer, or service technician before attempting to assemble the product to avoid personal injury or damage to the product. Each user should ask his doctor or therapist to determine the appropriate variant of regulation and method of use.

The walker is not intended to be guided by the user while sitting on it.

A physician / therapist should be present when adjusting the height of the walker to provide support for the user and to properly tighten the screws.

When the walker is unfolded always make sure that the tube under the seat is locked and properly attached to the frame. Make sure the height adjustment knobs are locked so that the main support structure will not move under pressure. Failure to follow the above recommendations may lead to injuries.

Before use, make sure that ALL the adjustment elements of the handles and the seat are secure and that the wheels and moving parts of the product are in good condition.

During use, ALL the wheels must be in contact with the ground AT ALL TIMES to ensure the stability of the walker.

Failure to do so may cause the walker to tip over and injure the user.

CAUTION: When using and operating the walker, and when folding it and adjusting the mechanisms, there may be a risk of entrapment and / or pinching of body parts of the user / accompanying person in openings / gaps between the components. These steps should be performed with extreme caution. After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully.

ATTENTION:

When a product-related "serious incident" occurs, which directly or indirectly led, or could lead to, any of the following events:

- death of a patient, user or other person or
- temporary or permanent deterioration in the health of a patient, user or other person, or
- a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, consult a healthcare professional.

NOTE: In the event of pain, great discomfort, or an allergic reaction, please contact a healthcare professional as soon as possible.

WARNINGS REGARDING FOLDING AND ASSEMBLY

After assembling the walker, make sure that the seat is locked and that both handle tubes are attached with the nuts in the main support frame. Also make sure that both handles are at the same height and do not move.

WARNINGS FOR INSTALLATION

Make sure the seat tube is securely fastened to the lock and the copper pins slide out of the appropriate holes. BEFORE use, always check that the walker and its components are securely locked in place.

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing, or resulting from failure to comply with the recommendations contained in this manual.

ATTENTION: it is forbidden to use the product other than in accordance with its intended purpose

CAUTION: There may be a risk of tipping over if the product is used improperly. Please follow the boarding / dismounting / moving instructions.

INTENDED USE: The product is intended to support the movement of people with reduced walking ability. The walker is designed for one person

CONTRAINDICATIONS: Physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the walker.

PRODUCT FEATURES:

- light, durable, folded aluminum structure
- folding / unfolding with one hand for easy transport and storage
- large cast swivel front wheels with a heavy duty fork to facilitate movement in the field
- reflectors on the frame in the front part, ensuring safety
- comfortable backrest, seat and large shopping bag
- non-slip comfortable handles
- in addition to the brake in the handles, the walker also has parking brakes
- 4 levels of height adjustment

INCLUDED: ROLLATOR, BAG, USER MANUAL

UNFOLDING

- After unpacking the product, check that the walker is undamaged. (Fig.1)
- Position the walker and unfold it with the handles facing outwards. (Fig. 2)
- Press down on the seat until the latches engage. (Fig. 3)
- Pull the brake down to block the walker. (Fig. 4)
- Adjust the height of the backrest and then tighten the screws properly. (Fig. 5)
- Adjust the height of the armrest and then tighten the screws properly. (Fig. 6)
- The height of the handles can be set between 80cm and 90cm. (Fig.7)

FOLDING

- To fold the walker, lift the strap on the seat. (Fig.8)

Fig.1

Fig.2

Fig.3

Fig.4

Fig.5

Fig.6

Fig.7

Fig.8

Technical specifications

Maximum width: 61 cm
Length: 73 cm
Height adjustment: 84-94 cm (4 steps every 2.5 cm)
Folded thickness: 19 cm
The width between the handles: 42 cm
Seat width: 42 cm
Seat height: 50 cm
Front swivel wheel diameter: 25 cm
Rear non-steer wheel diameter: 20 cm
Weight: 8.4 kg
Maximum load: 136 kg

This sign indicates the maximum weight of the user.

STORAGE:

The product should be stored in a dry place, not exposed to direct sunlight.

MAINTENANCE

- Please ensure that all attachments are always secured.
- Check all mounting bolts, tighten if necessary.
- Replace any damaged or damaged parts immediately.

WHEEL MAINTENANCE

- Wheels should be cleaned with warm water and a mild detergent. Pat dry with a dry cloth.
- Lubricate periodically with 3-in-1 grease if the wheels begin to squeak.

UTILIZATION

Municipal waste should be disposed of in accordance with the applicable disposal conditions.

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