AT51112

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ROLLATOR USER MANUAL

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The purpose of the rollator.

The lightweight AT51112 4-wheel rollator is designed to provide support to people who cannot or do not feel confident enough to walk without some support. The rollator is intended for indoor and outdoor use. The device can be folded for storage and transport. The product is additionally equipped with a seat and a bag.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

General safety information

1. Do not exceed the maximum user weight of 136 kg.
2. The rollator must not be used on stairs and escalators.
3. Do not attempt to push the rollator while a person is sitting on the seat.
4. The seat must not be used to transport people or goods.
5. Do not use the seat without the folded backrest.
6. Do not sit on the seat when the rollator is on an inclined surface.
7. The rollator is for walking support only.
8. The rollator may only be used with specified accessories and replacement parts.
9. Items can only be transported in a bag. Do not hang heavy objects on the handles of the rollator, as this may impair the stability of the rollator.

ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms associated with the use of the product please consult your healthcare professional.

ATTENTION:

In the event of a product-related „serious incident“ that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- c) a serious threat to public health

this „serious incident“ should be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE: The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

ATTENTION: it is forbidden to use the product in a way other than for its intended purpose!

NOTE: If the product is used incorrectly, there may be a risk of tipping over. Please follow the getting on/getting off/moving instructions. After completing the adjustment, stabilize the position by carefully tightening the nuts/screws.

General precautions

1. Make sure the rollator is properly unfolded before use.
2. Check the brakes before use.
3. Make sure all parts are in good condition and that moving parts are functional.
4. Improper use of the rollator may cause serious injury to the user and/or cause damage to the frame or wheels of the rollator.
5. Do not use the rollator to walk backwards.
6. Use caution when going over curbs and other obstacles, and when using the rollator on

sloped, uneven, or slippery surfaces. Otherwise, serious injury to the user may result.

7. Do not make any adjustments while using the rollator.
8. Be careful if there are heavy items in the bag (max. weight 5 kg).
9. All wheels must always be in contact with the ground.
10. Block the brakes before using the seat.
11. Block the brakes when using the rollator in a stationary position.

Product unpacking warning

1. Please check all parts carefully before assembling the rollator.
2. Do not assemble the rollator if any parts are damaged.
3. Do not assemble the rollator if any of the parts listed in the delivery range are missing.
4. When unpacking the parts, make sure that the handles and brake cables are not tangled, as this may adversely affect the operation of the assembled brakes.

DELIVERY RANGE

Rollator: main frame, backrest, rear wheels with handles, set of front wheels, cane holder, instruction manual

Assembly and height adjustment

1. Make sure all wheels are in good condition and rotate freely.
2. Make sure that the bag is properly attached and that all fastening latches are closed.
3. To assemble the rollator, the front and rear wheels must be secured using the push-button clips at the rear wheels and the „C” clips at the front wheels. All items should be placed at the same height. (fig.1 and 2)
4. Place the handles in the frame and then tighten the knob. (fig.3)
5. The height of the seat is adjusted by moving the wheels' mechanisms embedded in the frame and locking them in a fixed position with appropriate latches. Make sure that a click is heard when the clamps are locked, that all wheels are in contact with the ground, and that the rollator is stable.
6. To adjust the height of the handles, do the following:
 - a. Stand upright behind the rollator, relax your shoulders and lower your arms along your body, looking straight ahead.
 - b. The handles of the rollator should now be adjusted so that they are at the height of the wrists.
 - c. In this way, the user's arms will be bent at an angle of approximately 20-30 degrees when using the rollator.
 - d. Make sure the handle holes and the frame are aligned.
 - e. Insert the handles into the appropriately oriented holes in the frame, then tighten them firmly with the knobs.
7. The backrest should be placed in the appropriate holes (fig. 4)
8. Before using the rollator, check the brakes carefully. NOTE: The brakes have been adjusted by the manufacturer and, if correctly assembled, do not require readjustment before use. How to check the brakes: Block and release the parking brake ten times on each side. (The brake is locked by pushing the lever down and released by pulling it up.) The applied brake should immobilize the wheel and prevent it from moving. When released, the gap between the brake pad and the wheel should be 3-4 mm.
9. Make sure all other nuts, bolts and screws are tight.

1.

2.

3.

4.

Folding

All of the above steps must be performed in reverse order.


Sitting on the product

When using the AT51112 rollator to sit on it, strictly follow the precautions on page one, in particular:

1. Make sure the parking brake is blocked.
2. Do not use the seat without the backrest folded to the correct position.
3. Do not use the seat if the rollator is on a sloped or uneven surface.
4. Do not exceed the maximum user weight.
5. Do not use the rollator as a wheelchair.

Checking and using the brakes

1. Before using the rollator, always make sure that the brakes are fully functional.
2. Check the operation of the brakes by pulling up the brake levers as shown in Fig. 3 - the rollator should not move.
3. To block the parking brake, push the brake levers down. An audible click means the brake is in the correct position - the rollator should not move (see Fig. 4). Note that both levers must be in down position when using the parking brake. To release the parking brake, the levers must be moved up.
4. If the rear wheels of the rollator continue to move after blocking the parking brake or pulling up on the brake lever while walking, stop using the rollator immediately. It must not be used until the brakes have been checked and adjusted. See maintenance section.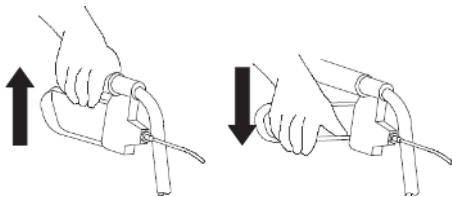

3

4

Cleaning



1. The rollator should be cleaned regularly by wiping it with a soft, damp cloth and then wiping it dry.
2. Do not allow large amounts of dirt to accumulate around the brake axles or on the wheels. Do not use strong cleaning agents or abrasive objects.

Maintenance

1. Regularly check rollator parts for wear and make sure all bolted connections are properly made.
2. Make sure the brakes are working properly by applying them while walking until the rollator stops.
3. Make sure that the parking brake is working properly by moving the levers to the position that causes it to block and attempting to move the rollator.
4. If using the brakes does not bring the wheels to a complete stop, the brakes need to be adjusted.

Adjusting the brakes (see Fig. 4)

1. There are two brake adjusters on both sides of the rollator - one in front of the brake lever and the other on the rear legs of the frame, behind the wheels.
2. To adjust the brake lever, loosen the lock nut (for this purpose, it may be necessary to use appropriate tools), turn the adjustment element counterclockwise to increase the brake pressure, and then tighten the lock nut again (see Fig. 4).
3. To adjust the brake on the rear legs of the frame, loosen the locknut (use of appropriate tools may be required), turn the adjuster counter-clockwise to increase brake pressure, then re-tighten the locknut. Proper adjustment of the brake mechanism requires a gap of no more than 3-4 mm between its lining and the wheel - these elements should also not touch each other.
4. Note: After changing the brake settings, test them before using the rollator.
5. If the adjustment does not increase brake pressure, do not use the rollator.

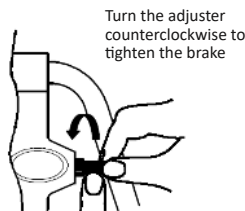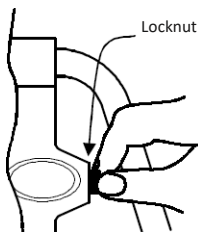5

SPECIFICATION

- Total width: 63 cm
- The distance between the handles: 45 cm
- Seat height: 53 - 69 cm
- Handle height adjustment range: 85 - 95 cm
- Rollator height: 66 cm
- Seat width: 32 cm
- Net product weight: 6.6 kg

Wheel diameter: 19 cm
Maximum bag load: 5 kg
Maximum user weight: 136 kg

**THIS MARK INDICATES THE
MAXIMUM USER WEIGHT**

Note: During the use and operation of the rollator, as well as when folding it and adjusting the mechanisms, there may be a risk of entrapment and/or pinching of the user's/assistant's body parts in the openings/gaps between the elements. These steps should be performed with particular care. After adjustment, stabilize the position by carefully tightening the nuts/bolts.

TARGET GROUP OF PATIENTS

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/size/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

STORAGE

The product should be stored in a dry, cool room, away from moisture and direct sunlight.

HOW TO DISPOSAL OF THE PRODUCT

The medical device can be disposed of as ordinary municipal waste.

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WARRANTY CARD

Model:
 Serial number:

Point of sale stamp
 and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
 03-068 Warsaw, ul. Zawiańska 43
 Tel. 22 518 36 00, Fax 22 518 36 30
 www.antar.net

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