## **AT51003**

**INSTRUKCJA OBSŁUGI CHODZIK  
USER MANUAL ROLLATOR  
BENUTZERHANDBUCH ROLLATOR  
UŽIVATELSKÝ MANUÁL CHODÍTKO  
NÁVOD NA POUŽITIE CHODÍTKO  
MANUEL DE L'UTILISATEUR ROLLATEUR  
HANDLEIDING ROLLATOR  
MANUAL DE USUARIO ANDADOR  
MANUALE D'USO ROLLATORE  
ANVÄNDARMANUAL ROLLATOR**

**MD**

**CE**

### Cechy produktu

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#### Product features

- The rollator is made of high-quality aluminum.
- The foldable frame makes it easier to transport.
- The rollator is equipped with swivel front and non-swivel rear wheels.
- The product is enriched with a seat and backrest, which allows you to rest in a sitting position.
- The rollator also has a convenient basket for essentials or small purchases.

#### WARNINGS

Before starting any operations, check the condition of the device. If the product works or looks abnormal, please contact the dealer where you purchased the product immediately. Antar Sp. Jawna is not responsible for improper use of the device.

#### General safety information

1. The maximum user weight of 120 kg must not be exceeded.
2. The rollator may not be used on traditional and escalator stairs.
3. Do not try to push the rollator when a person is sitting on the seat.
4. The seat must not be used for transporting people and goods.
5. The seat cannot be used without folded backrest.
6. Do not sit on the seat when the rollator is on a sloped surface.
7. The rollator is only intended as a walking aid.
8. The rollator may only be used with certain accessories and replacement parts.
9. Items may only be transported in the basket. Do not hang heavy objects on the handles of the rollator, as this may reduce the stability of the rollator.

#### General precautions

1. Make sure the rollator has been unfolded before use.
2. Before use, check that the brakes work.
3. Make sure all parts are in good condition and moving parts are operational.
4. You must not ride while sitting on the seat. Improper use of the rollator may result in serious injury to the user and / or damage the frame or wheels of the rollator.
5. The rollator must not be used for walking backwards.
6. Use caution when passing curbs and other obstacles and when using the rollator on sloped, uneven or slippery surfaces. Failure to do so may result in serious personal injury.
7. While the rollator is in use, it is not allowed to make any adjustments.
8. Be careful when there are heavy items in the basket (max weight 5 kg).
9. All wheels must always be in contact with the ground.

#### PREPARATION FOR USE

1. Product packed in folded position, basket disconnected. After removing from the packaging, the rollator should be unfolded by blocking the horizontal expansion hinges.
2. Then unfold the seat, install the handles and basket.
3. Adjust the height of the handles.

#### Attention:

When using and handling the rollator, and when folding it and adjusting the mechanisms, there may be a risk of entrapment and / or pinching of body parts of the user / accompanying person in openings / in gaps between the elements. These steps should be performed with extreme caution. After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully. To get in and out of the rollator safely, you need to apply the road wheel brakes.

#### USE OF BRAKES

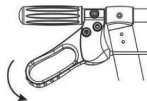
Fig. 1 Parking brake locked.

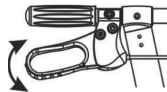
Fig. 2 Parking brake unlocked.

**STORAGE AND DISPOSAL**

Keep the rollator away from light sources, high temperature and moisture. Avoid exposing the rollator to direct sunlight.

Dispose of the rollator in accordance with the applicable environmental protection and recycling regulations.

**GENERAL PRECAUTIONS FOR USE**

Before use, always make sure that the handles are at the correct height and that they are properly fastened. To avoid the risk of falling, the rollator must not be used on uneven or sloped surfaces. Check the condition of the wheels and brakes. Finally, check the overall stability of the frame.

After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully.

**MAINTENANCE AND CLEANING**

After each use, remove dirt with mild detergents, then allow rollator to dry. Do not use sharp, rubbing sponges. Periodically check the operation of the brakes, the condition of the seals of the fastening elements and the condition of the welds.

**Technical Specifications**

Max user weight 120 kg

seat thickness 4 cm

width between handles 50 cm

seat width 36 cm

seat height from the ground 57 cm

height adjustment range of the handles 78– 89 cm

weight 8.80 kg

This sign indicates the maximum weight of the user.

**INTENDED USE:** The product is intended to support the movement of people with reduced walking ability. The rollator is designed for one person

**CONTRAINDICATIONS:** physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the rollator

**WARNING:**

When a product-related "serious incident" occurs, which directly or indirectly led, or could lead to, any of the following events:

- death of a patient, user or other person or
- temporary or permanent deterioration in the health of a patient, user or other person, or
- a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

**WARNING:**

In the event of pain, allergic reactions or other disturbing symptoms, unclear for the user, related to the use of a medical device, consult a healthcare professional.

**ATTENTION:** it is forbidden to use the product other than in accordance with its intended purpose

**NOTE:** Before use, check that all components are properly assembled.

**ATTENTION:** The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing, or resulting from failure to comply with the recommendations contained in this manual.

**INCLUDED**

Rollator, INSTRUCTIONS

**TARGET PATIENT GROUP**

The healthcare professional shall indicate the use of the device to adults and children on his/her own responsibility, taking into account the available variants/sizes/necessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

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### WARRANTY CARD

Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to AN TAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 AN TAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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