**AT52313**

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## ENG

Before using your power wheelchair for the first time, read and comply with all information, warnings and cautions in this manual. If there is anything you do not understand in this manual, or if you need additional assistance with installation or operation, please contact your retailer or manufacturer.

The safe use of the product depends on the observance of the information, cautions and warnings contained in this manual. We are not responsible for any damage and/or injury resulting from using the product in an unsafe manner or from failure to follow the instructions, cautions and warnings contained in this manual.

**CAUTION** Failure to follow the warnings and cautions contained in this manual may result in personal injury and damage to the electric wheelchair.

### 1. INTRODUCTION

For your own safety, please read and strictly follow these instructions for use when using your power wheelchair for the first time. These instructions are extremely important. Understanding the instructions is essential for the safe operation of the wheelchair.

#### GENERAL WARNINGS.

- Handle curbs carefully and make turns on hills.
- Steep and uneven surfaces should be avoided as this may cause the wheelchair to tip over.
- Avoid very soft surfaces as the wheels may sink in.
- Move the wheelchair smoothly in all conditions.
- Always keep both feet on the footrests and ensure that clothing is properly stowed and does not block the wheels.
- Always drive carefully on inclines.
- Always keep both hands on the push handles to control the wheelchair correctly (assistant only).
- Always remember to keep your balance.
- Check the wheelchair components regularly and always follow the instructions in this manual during and after the warranty period.
- Keep the wheelchair away from sources of fire
- Under no circumstances should the wheelchair be overloaded.
- Do not hang any heavy objects on the back of the stroller, as this may affect the stability of the stroller, especially when climbing hills.
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not raise the front wheels while driving, as this may cause a fall and serious injury.
- Do not get into the wheelchair while it is moving.
- Do not move the wheelchair too fast and do not make sudden turns.
- Do not stand on the footrests when getting in or out of the wheelchair.
- Do not attempt to lift the wheelchair with moving (removable) parts. Lifting with the moving parts of the wheelchair can lead to injury to the user and/or the caregiver or damage to the wheelchair.
- Do not use the stroller with the backrest folded down. This changes the center of gravity of the wheelchair and may cause the wheelchair to tip backwards. The backrest should only be folded when storing or transporting the stroller.

#### COMMENTS:

The power wheelchair is not intended to be used in the following cases:

- (1) In cases prohibited in this manual, such as driving on steep gradients, over obstacles that are too large, on a highway, in motor vehicle lanes.
- (2) In some places or on certain surfaces, e.g. a sloping surface covered with wet grass where the wheels can slip.
- (3) When the control system or other important parts need to be repaired.

**NOTE:** Inspect all parts for shipping damage. If you notice such damage, DO NOT use the product. More information from the manufacturer.

**NOTE:** If the product is used incorrectly, there may be a risk of tipping over. Please follow the boarding/dismounting/moving instructions.

#### ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- c) a serious threat to public health

this "serious incident" should be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

#### ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult a healthcare professional.

**ATTENTION: it is forbidden to use the product in a way other than for its intended purpose!**

**Application:** The product is intended for comfortable transport of people with limited or no walking ability. The wheelchair is designed to transport one person

**Contraindications:** physical or mental limitations (e.g. visual impairment) that prevent safe handling of the wheelchair.

## STORAGE

The product should be stored in a dry, cool room, away from moisture and direct sunlight.

## TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/size/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

## 2. PRODUCT DESCRIPTION

The power wheelchair consists of several parts: The frame consists of a component that can be folded for convenient storage or transport. The electronic control system consists of a control panel, a battery, two motors, a set of drive mechanisms and swivel wheels. Under the seat there is a 24 V 12 AH battery and its overload protection. The battery charging socket is located on the back of the control panel. The wheelchair is equipped with a brake lever on the drive mechanism. Appropriate operation of the lever causes the electric wheelchair to stop. In addition, the product is equipped with an anti-tip "mustache".

## 3. TECHNICAL PARAMETERS

Overall size (length x width x height): 1090mm x 640mm x 1350mm

Seat height: 540mm

Seat width: 430mm

Seat depth: 440mm

Armrest height: 200mm

Back height: 800mm

Battery weight: 2.5 kg

Weight of the wheelchair with the battery: 37 kg

Maximum speed: 6km/h

Braking distance at 6km/h:  $\leq 1500$ mm

Maximum turning radius:  $\leq 1200$ mm

Maximum load: 120kg

Range: 20km

Static stability:  $\geq 9^\circ$

Dynamic stability:  $\geq 6^\circ$

Permissible elevation:  $6^\circ$

Motors: 24V / 250W x 2

Battery: 24V/120Ah x 1

Control panel maximum output current: 50A

Battery output current: 2A

Front wheel: Tires outer diameter: 190mm

Rear wheel: Pneumatic tires outer diameter: 406mm

Overall width when folded: 390mm

## INFORMATION

This symbol indicates the user's maximum weight

### Troubleshooting

The electric wheelchair is equipped with a fault alarm system. In the event of damage to the wheelchair, the LED on the control panel will light up and an alarm will sound. This system allows fault detection according to Table 1.

If faults occur after previous faults have been rectified, please consult authorized customer service.

Table 1 Basic Troubleshooting

Alarm	Number	LED Cause
1	1 LED	Low battery
2	2 LEDS	Left motor fault
3	3 LEDS	Left brake fault
4	4 LEDS	Right motor fault
5	5 LEDS	Right brake fault
6	6 LEDS	Control panel overload
7	7 LEDS	Joystick fault
8	8 LEDS	Control panel fault
9	9 LEDS	Control panel fault

**Table 2 - To keep your electric wheelchair in good condition, it should be inspected before each use. In addition, wheelchair inspections should also be carried out on a weekly, monthly and semi-annual basis as shown in Table 2.**

	Every use	Weekly	Monthly	Every six months
Each part			<input type="radio"/>	
Turning, driving, settings, assembly, etc.		<input type="radio"/>		
Brakes	<input type="radio"/>			
Cable connections		<input type="radio"/>		
Charging the battery	<input type="radio"/>			
Operation of the front wheels		<input type="radio"/>		
Pneumatic rear wheels		<input type="radio"/>		
Tire wear			<input type="radio"/>	
Tire damage	<input type="radio"/>			
Wear of handles, backrest and seat	<input type="radio"/>			
Engines				<input type="radio"/>
Control Panel		<input type="radio"/>		
Purity	<input type="radio"/>			

#### 4. ASSEMBLY

Remove the electric wheelchair from the packaging, unfold the wheelchair by pulling the backrest outwards, then lift the backrest using the handles at the back of the backrest until the locking mechanism engages. Unfold the frame of the wheelchair towards the outside and press down on both sides of the seat until it is completely unfolded. To fit the upper part of the backrest, place the two tubes in place on either side of the seat base. (see fig. 1, 2 ).

2 Put the cushions on the seat and backrest (see fig. 3).

3. loosen the screws located at the control panel bracket and position it at the armrest . Then install the control unit by tightening the screws (see fig. 4). It is possible to mount the control unit on both the right and left sides.

Connect the control unit's cable plugs of corresponding description to the battery, fix the cables to the frame with plastic cable ties (see Fig. 5 and 6). .

5 Fit the footrests to the tubes on the left and right sides of the frame (see fig. 7). Turn the footrests forward to lock them in the extended position, fit the anti-tip wheels (see Figures 7 and 8).

6. fix the straps with nylon tape (see fig. 9).

7. adjust the height of the footrest, unscrew the locknuts on both sides of the footrest, remove the bolts (see fig. 10), move up or down the component on which the footrest is fixed, adjust to the correct height, then insert the bolts and tighten the nuts.


#### 5. OPERATION

##### Control panel

The control panel is a key component of the power wheelchair. The panel contains all the electric elements to operate the wheelchair (see Fig. 11). The control panel is located on one of the armrests and is connected to the batteries along with the motors.

The control panel consists of the following parts:

1. Joystick
2. Battery indicator
3. On/off button
4. Speed indicator
5. Acceleration button
6. Release button
7. Horn
8. Charger socket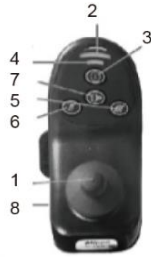


11

**joystick**

The joystick is mainly used to control the speed and direction of the wheelchair. The farther the joystick is moved from the center position, the faster it moves wheelchair. When released, the joystick automatically returns to the center position and the brake automatically applies.

**Acceleration/deceleration buttons**

When powered on, the speed indicator will indicate the maximum speed of the wheelchair. The maximum speed range indicated by the number of lights can be adjusted by the user. Each time the acceleration (or deceleration) button is pressed, the speed will increase (or decrease).

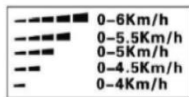12

**Horn**

After pressing the button, the horn will sound.

**Brake**

To brake the electric wheelchair, pull the brake lever back so that it is pressed against the wheels, which will bring the wheelchair to a stop. When the power wheelchair is in motion (with or without power), ensure that the lever is not pressed against the wheels by pushing the lever forward (see figure 13).

13

**Seat belt**

For your safety, fasten your seat belt whenever you use your wheelchair. Push the belt in until you hear a click (see figure 14).

14

15

**Position leverage**

Pull the lever to adjust the angle of inclination and return to the standard position (see figure 15).

**Footrests**

The wheelchair is equipped with two footrests with four-stage angle adjustment and the ability to adjust the appropriate height (see Figures 16 and 17).

16

17

## 6. Precautions

Make sure the control panel is properly installed and the joystick is in the center vertical position.

In order to prevent any damage to health, after sitting down, fasten the seat belts, rest your body on the backrest and raise your head. After switching on the wheelchair, first check that the maximum speed indicated by the speed indicator on the control panel is suitable for the user. If not, adjust it using the acceleration or deceleration button on the control panel. To be on the safe side, we recommend using a slower speed and then adjusting it by increasing it step by step.

If the wheelchair moves voluntarily, release the joystick immediately, which will stop the wheelchair if the joystick is functional.

**Warning:** Keep your feet on the footrest only;


**Warning:** Do not extend your arms beyond the armrests while driving.

**Warning:** Before using the wheelchair, make sure it is locked with the brakes.

## 7. Folding/unfolding and storing the wheelchair

**The wheelchair must be folded for transport and storage as follows:**

1. place the wheelchair on the ground. Disconnect the power supply.
  2. lift the footrest and rotate outwards by 90° (see figure 18).
  3. tilt the footrest frame section outwards, rotate 90° and pull (see figure 19).
  4. press the backrest folding lock on both sides of the handle (see figure 20).
- Fold the backrest and frame of the pram by lifting upwards by the centre of the seat (see figure 21).

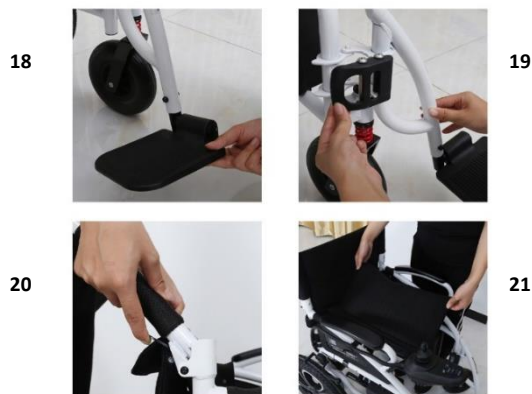## 8. Charging the battery

The battery can be quickly and easily recharged using the separately supplied electric wheelchair charger.

**ATTENTION!** The wheelchair battery should only be charged with the supplied charger. Do not use other battery chargers, such as car chargers.

Charge the battery with the separately supplied charger:

- Make sure that the control panel is turned off and that the brake lever is applied.
- Connect the three-pole plug to the three-pole socket on the control panel (see figure 22).
- Connect the charger plug to the power socket.
- Charging is signaled by the LED turning red. When the battery is fully charged, the LED turns green.
- The recommended charging time is 8-12 hours.
- When charging is complete, disconnect the charger plug from the electrical outlet and then disconnect it from the plug on the control panel. Keep the charger, cables and plugs together in the pocket behind the backrest.

## 9. Overload protection

When the motors are overloaded, the overload protection will cut off the current to protect the motors and their electrical parts. The overload protection must be set by qualified specialists: unscrew the screws on the battery box, replace the backup fuse with a broken one, replace the battery box cover, tighten the screws.

### 10. Seat belt

To ensure safety when using a power wheelchair, seat belts must be fastened.

#### 12. Cleaning and maintenance after use

- (1) Turn off the power (preferably disconnect all cable connections).
- (2) Keep the wheelchair away from children.
- (3) The wheelchair should be stored at normal temperature to prevent damage and prolong its service life.
- (4) Use a clean and soft cloth to clean the frame. The frame must be dried.

Scope of delivery: electric wheelchair, 3 pillows (2 for the backrest, 1 for the seat), two footrests, a belt for the footrests, a controller, anti-tip "mustache", instruction manual.

### HOW TO DISPOSE OF THE PRODUCT

The medical device is an electrical product, please follow the proper way to dispose of electrical and electronic equipment. NOTE: contains batteries.

### 13. WARRANTY

**1.1 Serial Number** To provide after-sales service and warranty, please enter the Vehicle Identification Number.

MODEL	AT52313
SERIAL NUMBER	

#### 1.2 Content of the warranty

1.2.1 Repairs are carried out free of charge if the problems are of a manufacturing nature or are related to a defective part during the warranty period (this type of repair will be referred to as a warranty repair in the following).

1.2.2 Warranty repairs may only be performed by ANTAR service.

1.2.3 ANTAR is not responsible for transport costs related to repair and replacement.

1.2.4 The replaced parts are the property of ANTAR.

#### 1.3 Warranty Duration and Covered Parts

Item	Warranty Duration
Electrical parts (motor, controller, charger)	12 months from the date of purchase
Wheelchair frame	12 months from date of purchase
Battery	6 months from date of purchase

#### 1.4 Parts not covered by the warranty

The items listed below are not covered by the warranty:

1.4.1 Wear resulting from use and the passage of time (e.g. normal ageing and deterioration of paint, plastic parts, tyre wear, etc.);

1.4.2 Damage caused by lack of maintenance or improper handling and storage;

#### 1.5 Problems not covered by warranty

**Problems arising from the following reasons are not covered by the warranty:**

1.5.1 Improper storage;

1.5.2 Tampering to change or add components to the vehicle that are not approved by ANTAR;

1.5.4 Damage caused by the use of parts other than genuine ANTAR parts or other parts not recommended by ANTAR;

1.5.5 Excessive or improper use of the vehicle, such as running over curbs and overloading the wheelchair with passengers or goods;

1.5.6 Driving in places where the vehicle should not normally be turned on or used;

1.5.7 Surface damage or corrosion caused by environmental factors such as air fall (chemicals, tree sap, etc.), stones, hail, windstorms, lightning, floods, etc.;

1.5.8 Alteration of any materials or components not authorized by ANTAR (items should not be transported in a manner that may cause the wheelchair to become unbalanced);

1.5.9 The wheelchair must not be pulled or pushed by another vehicle when there is a user in the wheelchair or the rear wheels are not in freewheel mode; and without checking its structure.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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