AT52322

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Please read this owner's manual before using the wheelchair. Improper use of the wheelchair could result in harm or injury. If you do not understand the manual, contact your provider for a complete explanation of the instructions, cautions and warnings. Otherwise, injury or damage may result.

1.0 PRE-RIDE SAFETY CHECK

We recommend that you perform a safety check before each use to make sure your wheelchair operates smoothly and safely.

Perform the following inspections prior to using your wheelchair:

- Check the conditions of the tires. Make sure they are not damaged or excessively worn.
- Check all brakes. Make sure they are in working order.
- Check the seat. Make sure it is not excessively worn.

If you discover a problem, contact your authorized provider for assistance.

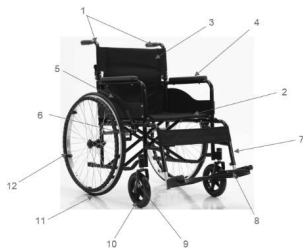
WARNING! Never exceed the maximum load.

2.0 SPECIFICATIONS

Product Weight: 14 kg
 Overall Length: 77~102 cm
 Overall Height: 74~87.5 cm
 Overall Width: 68 cm
 Seat Width: 47 cm
 Seat Depth: 42 cm
 Seat Height: 48 cm
 Back Height: 44 cm
 Front Castor: 20 cm PVC
 Rear Wheel: 59 cm RUBBER
 Folded Down Backrest: Yes
 Flip Up Armrest: No
 Detachable Footrest: Yes
 Footrest Height: Adjustable

3.0 WHEELCHAIR COMPONENTS

1. Handle Grips
2. Seat Upholstery
3. Back Upholstery
4. Armrest Pad
5. Side Panel
6. Hand Brake
7. Detachable Footrest
8. Footplate
9. Front Fork
10. Front Castor
11. Rear Wheel
12. Hand Rim

4.0 GENERAL SAFETY – WARNINGS FOR SAFE USE

WARNING! Failure to follow the instructions which are described in this user guide may result in damage to the wheelchair or body injuries to the user.

WARNING! Avoid using your wheelchair on roads.

WARNING! Do not use your wheelchair on sand, rough terrain or wet and slippery surfaces.

WARNING! Do not use the wheelchair if it is broken or not functioning correctly.

WARNING! The wheelchair is not suitable for use as a seat in a motor vehicle. The wheelchair could fail in a vehicle impact, leading to injury of the wheelchair occupant or others.

WARNING! Before leaning or reaching forward, always push your bottom to the back of the chair and face the castors forwards.

WARNING! Do not use hand brake to slow your descent down an incline. Abruptly changing directions while going down an incline could cause instability or possible tipping over.

WARNING! Never connect anything to the wheels. This may cause damage to the wheelchair and affects the balance of the wheelchair and may injure the user.

WARNING! Always engage the brakes when you are in or on a lift and when you want to get off.

WARNING! Never exceed the maximum load and never use the wheelchair to transport more than one person.

WARNING! For suitable protection from potential obstruction, the footrests should have a minimum clearance of 7 cm from the ground.

WARNING! Never stand on the footplates. This will cause the wheelchair to tip over and injure the user (Fig 4.0.1).

Fig. 4.0.1

WARNING! Always place the wheelchair on a stable, flat surface and engage the brakes before transferring to or from the wheelchair.

WARNING! Unauthorized modification and or use of parts not supplied by the provider will invalidate the warranty of this wheelchair and may lead to injury to the user or damage to the wheelchair.

Destiny

The wheelchair is intended for the transport of a person with reduced mobility. The trolley is designed to transport one person.

Contraindications

physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the product.

TARGET GROUP OF PATIENTS

The healthcare professional indicates the use of the product to adults and children at their own risk, taking into account the available variants / sizes / necessary functions / size and indications, taking into account the information provided by the manufacturer.

5.0 OPERATING INSTRUCTION

5.1 UNFOLDING WHEELCHAIR

1. Stand beside the wheelchairs
2. Hold both seat tubes and move them apart.
3. Push both seat tubes downwards to unfold the wheelchair. (Fig 5.1.1)
4. Lock seat tubes in place and check if seat upholstery is flat.
5. Replace the footrests, put down the footplates, and unfold the backrest. Wheelchair is now ready for using.

Fig. 5.1.1

5.2 FOLDING WHEELCHAIR

1. Folded down backrest, and fold the footplate and remove the footrests.
2. Stand beside the wheelchair
3. Hold the edges of the seat in the middle of the seat upholstery and pull the it straight up. (Fig 5.2.1)

Fig. 5.2.1

5.3 FOLDED DOWN BACKREST

1. The backrest is quick and easy to fold for storage and when transporting the wheelchair.
2. Stand behind the wheelchair and squeeze the two black levers (Fig 5.3.1) together and at the same time fold down the backrest.
3. The folding mechanism is now unlocked and you can fully fold down the backrest (Fig 5.3.2).
4. To raise the backrest from the folded position, hold both handles and lift up together to click lock (Fig 5.3.3).

Fig. 5.3.1

Fig. 5.3.2

Fig. 5.3.3

5.4 REMOVING THE FOOTREST

1. To remove footrest, stand at front of the wheelchair. Pull the inserter upwards (Fig 5.4.1).
2. Pull the footrests outwards and remove them off the brackets (Fig 5.4.2).

Rys. 5.4.1

Rys. 5.4.2

5.5 INSTALLING THE FOOTREST

1. To install footrest, place the footrest into the wheelchair brackets (Fig 5.5.1). Push it inward.
2. Put the inserter into the hole to fix the footrest (Fig 5.5.2).
3. Check the footrest is securely fixed (Fig 5.5.3).

Fig. 5.5.1

Fig. 5.5.2

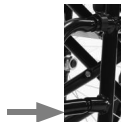
Fig. 5.5.3

5.6 FOLDING THE FOOTPLATE

1. Hold the front of the footplate (Fig 5.6.1).
2. Fold the footplate up, in the direction of the footrest (Fig 5.6.2).

Fig. 5.6.1

Fig. 5.6.2

5.7 ADJUSTING FOOTREST HEIGHT

1. Loosen the nut with a wrench and move the footplate bracket down and adjust the footrest to the required height. (Fig 5.7.1)
 2. Then use a wrench to tight the nut fully.
- Ensure two side footplates are at the same height

Fig. 5.7.1

5.8 HAND BRAKE

Engage the hand brake on both wheels before entering the chair, leaving the chair, leaning forward in the chair, or while being transported in an elevator or a wheelchair lift. Check to make sure that the footrests do not unlock the hand brakes when they are swung back. When adjusted correctly, the hand brake shoe should be embedded into the tire at least 1/8" when engaged in the locked position. Hand brakes are activated by pushing the handle completely forward. To adjust brakes, loosen the nut on the carriage bolt. Slide the clamp toward the rear wheel until the brake shoe is embedded at least 1/8 inch into the tire when the handle is engaged in the locked position. Tighten the nut and bolt to secure in position. Inspect for correct locking action BEFORE actual use. (Fig 5.8.1)

Fig. 5.8.1

5.9 Using the seat belt on your wheelchair

Your wheelchair is equipped with a seat belt which you can use the following way:

1. When in the wheelchair, you can fasten the belt by clicking the two loose ends together.(Fig 5.9.1)
2. If the belt is to loose or to tight you can make it more secure or looser.
3. To release the seat belt, you press on the lock plates to unlock.

Fig. 5.9.1

5.10 CAUTION:

Stability - The wheelchair is designed to be safe and stable as long as you maintain your center of gravity in the chair. Never reach for objects that require you to shift in the seat or bend over out of the chair. Whenever reaching for objects that require you to lean, make sure the front casters are pointing in a forward position, which extends the wheel base and will help stabilize the chair.

Ramps - Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. You and your attendant should first consult a qualified professional for correct techniques on ramp negotiation. If it becomes necessary to stop on an incline, avoid sudden, abrupt forward movement when you resume propelling the chair. Because of the backward angle of the chair, sudden abrupt movement could cause instability. Attempting to use the wheel lock brakes could result in accidental locking, veering or tipping to one side, or suddenly stopping. Avoid changing direction when going down an incline, as this could cause instability.

6.0 MAINTENANCE & MONTHLY CHECKS

It is highly recommended that the wheelchair be taken to a qualified provider annually for a thorough inspection and maintenance/repair service.

General Care - Clean all metal parts with auto wax or similar products. Dry the chair immediately if exposed to moisture. Do not use abrasive cleanser or material that will scratch the finish off the chair.

Monthly Checks:

Inspect the entire wheelchair for broken parts and loose fasteners. Replace or tighten as needed.

Front and Rear Tires - Solid tires require minimal maintenance, but should occasionally be cleaned with a damp cloth. Replace the tires if they become cracked or severely worn.

Front Caster and Rear Wheels - Check that the wheels are not warped and the spokes are not broken. Also assure that the bearings are in good working condition and the wheels spin freely.

Fork Forks - Verify that the forks swivel freely without binding. If binding occurs, check the bearings and clean the fork stem housing. If the nut is too loose, the caster will flutter or shimmy. If it is too tight, the chair will be difficult to steer.

Wheels - The large rear wheels should be checked for correct tightness. If the wheel wobbles or is loose, the bearings may be worn. Take the wheelchair to your local provider for service.

Frame - Check that the frame is not bent or dented. Check the tubing for cracks and rust. If such issues are found, take the wheelchair to your local service provider.

Clean and lightly oil the three rear axes on each side of the frame to prevent rust. Plastic caps should be placed over the axes while not in use.

7.0 WARRANTY

This warranty is extended only to the original purchaser. Your original receipt will be necessary as proof of purchase before any warranty performances are rendered.

The chair was built to exacting standards and carefully inspected prior to shipment. Product is warranted to be free of defects in materials and workmanship for a period of one year from date of purchase. Provider warrants the side frames and crossbraces for the lifetime of the original consumer purchaser. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of provider.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, including but not limited to rubber accessories, castors, grips and upholstery, armrest pad, which are subject to normal wear and need periodic replacement.

This warranty shall not apply to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstance beyond provider's control, such as improper operation, maintenance or storage.

If you have a question about your provider product or this warranty, please contact the provider.

WARNING:

In case a serious incident that has occurred in relation to the medical device, that directly or indirectly led, might have led or might lead to any of the following:

- a) the death of a patient, user or other person,
- b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- c) a serious public health threat;

the above mentioned serious incident should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established;

WARNING:

In case any pain, allergic reaction or other worrying symptoms, unclear for the user which have occurred in relation to the medical device, the user should consult a healthcare professional;

NOTE: Inspect all parts for damage caused in transit. If any such damage is found, DO NOT use the product. More information from the manufacturer.

ATTENTION: it is forbidden to use the product other than in accordance with its intended purpose

NOTE: Before use, check that all components are properly assembled.

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing, or resulting from failure to comply with the recommendations contained in this manual.

CAUTION: There may be a risk of tipping over if the product is used improperly. Please follow the boarding / dismantling / moving instructions.

CAUTION: When using and handling the product, as well as when assembling and adjusting the mechanisms, there may be a risk of entrapment and / or pinching of body parts of the user / accompanying person in openings / gaps between the elements. These steps should be performed with extreme caution. After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully

INCLUDED

Wheelchair, footrests, calf support belt, user manual.

SHelf LIFE

Based on the tests performed, the manufacturer indicates a 5-year shelf life of the medical device.

This symbol means: "Maximum user weight"

SAFE DISPOSAL OF THE MEDICAL DEVICE

When the device has been withdrawn from use it can be treated as municipal waste except for electric medical devices – in such case the electric medical device shall be disposed in the way described by the law for electrical and electronic waste disposal.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to AN TAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net e-mail: antar@antar.net

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