**AT52324 / AT52306**  
**INSTRUKCJA OBSŁUGI WÓZEK INWALIDZKI**  
**USER MANUAL WHEELCHAIR**  
**BEDIENUNGSANLEITUNG ROLLSTUHL**  
**UŽIVATELSKÁ PŘÍRUČKA INVALIDNÍ VOZÍK**  
**NÁVOD NA POUŽITIE INVALIDNÉHO VOZÍKA**  
**MANUEL D'INSTRUCTION FAUTEUIL ROULANT**  
**GEBRUIKSAANWIJZING ROLSTOEL**  
**MANUAL DE INSTRUCCIONES SILLA DE RUEDAS**  
**MANUALE DI ISTRUZIONI DELLA SEDIA A ROTELLE**  
**BRUKSANVISNING RULLSTOL**

**MD**

**CE**

PL

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ENG

**GENERAL WARNING**

Do not use the product before reading and understanding this manual. If you do not understand any warnings, cautions or instructions, please contact your healthcare professional or retailer to avoid damaging the product.

**CONTRAINDICATIONS**

Physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the product.

**PURPOSE OF THE PRODUCT**

The wheelchair is intended for the transport of persons with disabilities and persons with mobility difficulties. The product is intended for use by one person

**TARGET PATIENT GROUP**

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user independently or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

**TECHNICAL SPECIFICATIONS**

Frame: aluminium  
Dimensions L x W x H: 101 x 63 x 88,5 cm  
Folded dimensions: 74 x 28 x 68 cm  
Seat width: 46 cm  
Seat depth: 40 cm  
Backrest height: 42 cm  
Armrest height: 20 cm  
Height from the ground to the footrest: 18,5- 21 CM (2 steps of adjustment)  
Seat height: 47 cm  
Net weight: 10,7 kg  
Gross weight: 12,4 kg  
Maximum payload: 100 kg  
Rear cast wheels: 20"  
PVC front wheels: 6"

**TECHNICAL SPECIFICATIONS AT52306**

Frame: aluminium  
Dimensions l.w: 106 x 62 x 92 cm  
Dimensions when folded: 79.5 x 31 x 74 cm  
Seat width: 46 cm  
Seat depth: 40 cm  
Height from ground to seat: 53cm  
Height of backrest: 40cm  
Height of armrest: 23cm  
Height from the ground to the footrest: 19 - 21.5cm ( 2 degrees of adjustment)  
Seat height: 51.5cm  
Net weight: 12.4 kg  
Gross weight: 14.3 kg  
Maximum payload: 120 kg  
Rear cast wheels: 23"  
Front PVC wheels: 8"  
Wheelchair equipped with anti-tip wheels

**FEATURES OF THE PUSHCHAIR**

- safety belt
- additional rear pocket
- calf stabiliser strap
- foldable backrest
- quick release rear wheels



This symbol indicates the user's maximum weight

**REMARKS**

1. Remember to apply the brake even after a short stop. It is important to get into the habit of applying the brakes.
2. The wheelchair cannot brake suddenly, therefore you should not accelerate excessively while driving.
3. When making a sharp turn, slow down to avoid tipping over.
4. Before moving the wheelchair, make sure that the patient is keeping his feet on the footrest.
5. Before use, check that all elements are properly assembled.
6. Make sure that the brake is on when getting in and out of the wheelchair.
7. When getting off the wheelchair, do not stand on the footrest as the wheelchair will tilt forwards.
8. Do not put the wheelchair near any source of fire.
9. There may be a risk of tipping over if the product is not used properly. Please follow the boarding / dismantling / moving instructions.
10. During use and handling of the product, as well as during its assembly and adjustment of mechanisms, there may be a risk of entrapment and / or pinching of body parts of the user / accompanying person in openings / gaps between the elements. These steps should be performed with extreme caution. After completing the adjustment, stabilize the position by tightening the nuts / bolts carefully.
11. It is forbidden to use the product other than in accordance with its intended purpose

**SERVICE**


**FOLDING OUT A WHEELCHAIR WITH A FOLDING BACK**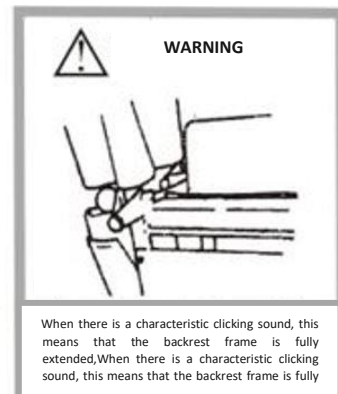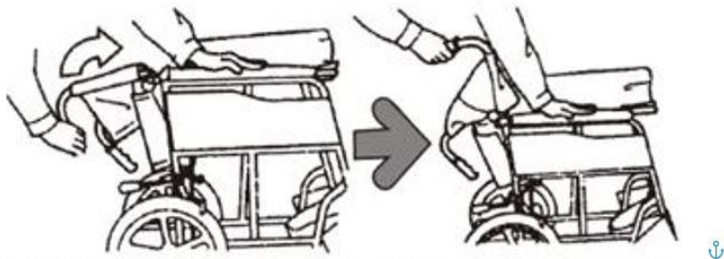

1. grasp the handle and lift it as indicated by the arrow to unfold the backrest.
2. while lifting the handle, press on the armrest.
3. press on the sides of the seat until the seat is fully unfolded.
4. lower the footrests down, fit the calf support belt.


#### FOOT ADJUSTMENT

Turn the footrest in the direction of the arrow and adjust its height. Then tighten the nut on the side of the footrest tube with the supplied wrench.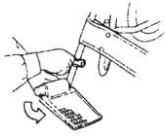

#### FOOT HEIGHT ADJUSTMENT

1. Using a wrench, slightly loosen the bolt where the footrest tube is bolted to the footrest, and then reattach it after completing the adjustment.
2. Rotate the footrest in the direction of the arrow. Then attach the footrest.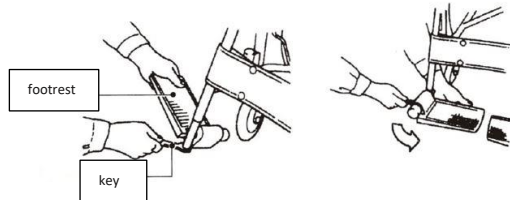

#### CAUTION


1. The footrest should be turned every time you get on or off the wheelchair.
2. Make sure that the footrest is at least 7 cm above the ground. If it is too low, the footrest will hit obstacles as it moves over uneven ground, resulting in an imbalance of the wheelchair.

#### USE THE BRAKE

The brake at the handle is only used to reduce speed. To lock the wheelchair, apply the brake lever. Be sure to apply both side brakes at the same time.


#### SIDE BRAKE

To lock the wheelchair, pull the levers of both side brakes backwards. Pushing them forward will release the brake.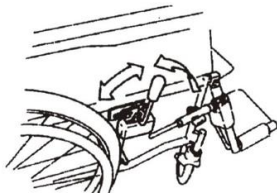

#### FOLDING THE WHEELCHAIR

1. Press on the backrest lock release clips, hold the handle with one hand and pull backwards with the other.
2. Then grasp the centre of the backrest and fold the wheelchair.
3. Lift the footrests upwards.

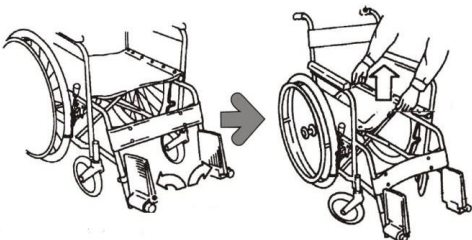#### CAUTION:

When a product-related "serious incident" occurs, which directly or indirectly led, could or could have led to any of the following events:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

This "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, please consult a healthcare professional.

#### CLEANING AND MAINTENANCE

- A. Wipe the main frame with a damp cloth and mild detergent. Do not use strong cleaning agents, caustic preparations and sharp / abrasive objects.
- B. To clean the seat cushions and the backrest, use lukewarm water with a mild detergent. Do not use strong cleaning agents and preparations.
- C. Leave to dry

#### STORAGE

Store the wheelchair in a dry and cool room. Avoid direct sunlight and moisture.

#### SCOPE OF DELIVERY

Wheelchair, calf stabilisation belt, instruction manual, footrest height adjustment key

#### HOW TO DISPOSE OF THE PRODUCT

The medical device can be disposed of as normal municipal waste.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 AN TAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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