AT52330

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After unpacking, ensure that all components have no visible mechanical damage, otherwise contact the manufacturer.

NOTES AND WARNINGS

SAFETY RULES

- Getting on and getting off the wheelchair should take place with the brakes adjusted and blocked.
- The brakes must not be used while the wheelchair is moving (only when stationary).
- To increase the stability of the product, make sure that the front wheels are aligned in the direction of travel before leaning forward.
- Do not lean forward excessively or lean to the sides of the wheelchair.
- Only use the wheelchair on surfaces where all four wheels touch the ground.
- Carry the product by grasping the frame structure. Never carry it by other parts of the product.
- It is not allowed to use the product if there is any visible or perceptible damage or if the degree of wear of particular parts of the wheelchair causes a potential risk of accident (wear of wheels, damage, damage to upholstery).
- When getting on and off of the wheelchair, do not stand on the footrests as this may cause the wheelchair to tip over.
- When driving over thresholds or other obstacles, extreme care should be taken when transferring all weight to the rear wheels. If there is any concern as to whether this manoeuvre can be carried out safely, it is imperative that a second person is requested to assist.
- When moving the wheelchair, take care that the fingers of your hands do not get between the spokes. This can lead to hand injury.
- Special care should be taken when driving on inclines. If you find it difficult to pass an incline safely, it is essential that you ask another person for help.
- Avoid contact of the product with fire. If textile, plastic and rubber parts come into direct contact with fire, they may ignite.

CAUTION: Incorrect or careless transfer of body weight to the rear wheels can lead to the wheelchair tipping over backwards, which is a very dangerous event. It can lead to serious injuries! If you are in any doubt as to whether it will be possible to safely pass an obstacle, it is imperative that you ask for help from another person!

ATTENTION: The manufacturer will not be held liable for damage caused by negligent maintenance, inadequate servicing or as a result of not following the other recommendations in this manual.

ATTENTION: It is forbidden to use the product in any other way than in accordance with its intended use!

ATTENTION: During use and handling of the product and when assembling and adjusting its mechanisms, there may be a danger of trapping and/or squeezing of parts of the user's/companion's body in openings/gaps between components. Perform these operations with particular care.

NOTE: There may be a risk of tipping over if the product is used incorrectly. Please follow the instructions for getting on/off/moving. Once the adjustment is complete, stabilise the position by tightening the nuts/bolts carefully.

CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

NOTE: It is forbidden to stand on the footrests as this can lead to a dangerous fall and injury.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

USE/INDICATIONS

The wheelchair is used for the individual transportation of disabled persons after spinal cord injury and with various lower limb injuries, neuromuscular diseases, after craniocerebral trauma or for geriatric patients. It is also designed for people who want to participate actively and comfortably in everyday life.

CONTRAINDICATIONS

A wheelchair should not be used by people who:

- Are unable to sit,
- Have impaired perception,
- Have severe balance disorders,
- Have lost both upper limbs,
- Have a visual impairment that will prevent safe use of the device.

MAINTENANCE / STORAGE

Periodically check the condition of the wheelchair and maintenance by:

- (a) checking the condition of the tyres - if they show severe wear, they should be replaced at the same time;
- (b) checking the tightness of the bolts of the connecting components;
- (c) cleaning the product from dirt with a damp cloth;
- (d) cleaning of upholstery with a damp cloth, possibly with a mild cleaning agent;
- e) it is important to keep all parts of the wheelchair clean and to store it properly in order to maintain the performance of the product. Storing the product in rooms with very high humidity (laundries, bathrooms) or in the open air will cause rapid loss of its useful properties. The product should be stored in a dry, cool room, away from direct sunlight.
- f) do not expose the wheelchair to extreme weather conditions (rain, sun, snow).
- g) do not use solvents, chlorine-containing agents, sharp brushes and other cleaning objects or corrosive cleaning agents.

OPERATION OF THE WHEELCHAIR

The wheelchair has a folding frame - it is folded by pulling the central part of the seat upwards and unfolded by pressing the sides of the seat downwards. Ensure that the sides of the seat rest accurately in the plastic catches. Inaccurate unfolding of the seat can lead to breakage of the catches and damage to the product.

fig. 1

fig. 2

fig. 3

fig. 3.1

Step 1: after unpacking, remove the wheelchair from the carton and check for scratches.

Step 2: using two hands, unfold the frame of the wheelchair, press down on the seat to fully unfold it

Step 3: lock the footrest as shown above, pressing it down from above so that it locks into place. (Fig.3 and 3.1)

fig. 4

fig. 5

fig. 6

Step 4: check that the parking brakes are working correctly

Step 5: fasten the upholstery with velcro fasteners

Step 6: attach the belt as illustrated above

fig. 7

fig. 8

fig. 9

Step 7: fasten the cushion with Velcro

Step 8: push the lever forward to lock the parking brake

Accessories: cushion, upholstery, belt, adjustment keys (fig.9)

FOOTREST ADJUSTMENT

fig. 10

fig. 11

To set the correct height of the footrest, unscrew the locking screw, adjust the height and screw it back in with the supplied spanners. (10 i 11)

CENTRE OF GRAVITY ADJUSTMENT

The pushchair has the possibility to adjust the centre of gravity by adjusting the rear and front wheel. We recommend that these are adjusted by a specialist.

fig. 12

fig. 13

FOLDING

To fold the wheelchair, all the steps described above must be carried out in reverse order.

MOVING IN A WHEELCHAIR

When moving in a wheelchair, the hands should be arranged so that the thumb lies along the top of the wheel and the fingers hold the bottom of the wheel. This ensures a firm grip and the ability to let go of the wheels quickly. Grasp the wheels at the height of the backrest and push them forward with a vigorous movement. A guide is fitted along the wheels to help you push the wheels firmly. After the movement, bring your hands back to the starting point so that they form a circle. This prevents the shoulders and arms from working during the return movement. Do not move the hands back to the starting point along the drive wheel. The wheelchair is designed so that the main load rests on the rear (large) wheels. In this way, the front (small) wheels remain relieved, preventing damage to them. However, the method of construction described above poses a certain risk of the product tipping over when moving on ramps, inclines or overcoming obstacles if safety rules are not observed. Therefore, please read the rules described in the instructions carefully.

OVERCOMING OBSTACLES:

- If for any reason you are unable to overcome an obstacle while moving in a wheelchair, please ask for assistance. Attempts to overcome an obstacle on your own may end up tipping the wheelchair over and causing injury.
- most users are able to overcome small obstacles on their own;
- descending a small threshold (kerb) - position yourself facing forwards, perpendicular to the axis of the threshold, ensure that the footrests do not overlap the ground and descend the threshold. Avoid leaning forward;
- when descending a large threshold (kerbstone), position yourself with your back perpendicular to the axis of the threshold, lean forwards, shift your centre of gravity forwards and then drive off the obstacle, controlling the speed of descent by supporting the drive wheels with your hands;
- in the case of steep obstacles or stairs, always ask for help;
- To ascend a threshold, transfer all body weight to the rear wheels and pass the step by pushing the drive wheels with your hands and balancing your body. The wheelchair should be positioned perpendicular to the obstacle. By balancing with the body, place the front wheels on the obstacle and then shift the body weight forward (by leaning forward) and push the drive wheels onto the obstacle. In the case of steep inclines or high obstacles, it is essential to ask for help.
- When driving on inclines, take extra care not to lose control of the wheelchair. Speed must be controlled at all times. If you are in any doubt as to whether you can safely pass an incline, it is essential that you ask for help from another person.

TECHNICAL SPECIFICATIONS AT52330: Seat width: 400 / 430 / 450 mm Seat height: 480 mm

Seat depth: 360 mm

Backrest height: 280 mm

Overall height: 790 mm

Overall length: 850 mm

Overall width: 620/630/640 mm Width when folded: 320/321/322mm

Rear pneumatic wheel diameter: 24" Recommended tire pressure: 2.5–3.0 bar

Front wheel diameter: 4"

Maximum load capacity: 136kg Carton size: 850*345*785 mm

Net weight: 11.7/11.8/11.9 kg

This symbol indicates the maximum weight of the user!

CLEANING

In order to prolong the life of the wheelchair, it is recommended to clean it periodically (preferably daily) especially if it is used in rainy or snowy weather. We recommend that you do not use the product in rain or snow. If rain is accidentally encountered while moving the wheelchair, it is recommended to find a shelter. If it is not possible to find shelter, head to your destination. Upholstery and frame are best cleaned with a damp cloth without strong detergents. Remove dirt on a regular basis e.g.: threads, hair etc. from moving parts of the wheelchair (wheels, footrests). Regularly remove dirt from textile components. In order to maintain the performance of the product it is extremely important to keep all its components clean, and the way it is stored.

DO NOT USE

- solvents
- toilet cleaners
- products containing chlorine
- sharp brushes and hard objects
- corrosive cleaning agents
- pressure or steam cleaners

FRAME

Wipe with a damp cloth and mild detergent.

UPHOLSTERY

Use warm soapy water to clean the upholstery. Upholstery cleaner/foam can also be used, but do not use strong detergents or solvents.

Keep the wheelchair away from sources of fire that may cause injury to the user and damage the product.

HOW TO DISPOSE OF THE PRODUCT

Once the device has been taken out of service, the medical device can be disposed of as normal municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. AN TAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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