AT51080

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USER MANUAL ROLLATOR

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

DO NOT assemble or use the product before reading and understanding these instructions. If you do not understand the warnings, cautions, or instructions, contact a health care professional, distributor, or technician before attempting to assemble the product to avoid injury or damage to the product.

When the rehabilitation rollator is unfolded always make sure that the tube under the seat is locked and properly connected to the frame. Check the brake cable after each height adjustment to avoid loosening the cable. Make sure that the knobs are locked, in such a way that the main support structure will not move due to pressure.

Note: Use only the bag included with the rollator, do not install third-party products, as you risk damaging the product.

USE/INDICATIONS

The product is designed to support the movement of people with limited walking ability. The rehabilitation rollator is intended for individual use. The product will find application in:

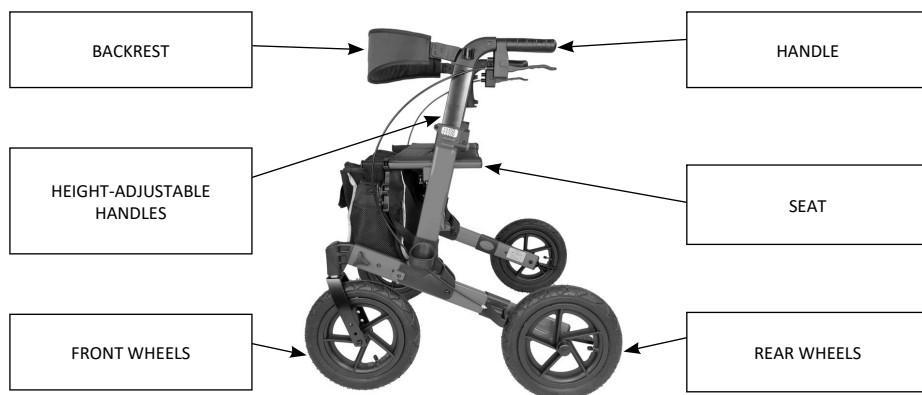
- Balance Training: The rehabilitation rollator enables patients to practice maintaining balance, which is important in the rehabilitation process after injuries or neurological disorders.
- Posture maintenance: Patients can use the rehabilitation rollator to train and maintain correct posture, particularly beneficial for individuals with postural defects or stability issues.
- Improving motor coordination: Exercises with a rehabilitation rollator can enhance motor coordination and movement control, vital for individuals with neurological conditions.
- Supporting rehabilitation after injuries: The rehabilitation rollator can be used in the rehabilitation process after lower limb injuries, helping patients gradually regain mobility.
- Increasing independence: For some patients, a rehabilitation rollator May serve as a tool to increase independence in daily life, enabling independent movement.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe operation.

- People with serious spinal stability problems or spinal injuries may not be suitable candidates for using a rehabilitation rollator, as it may increase the risk of further injury.
- In people with uncontrolled seizures or epilepsy, the use of a rehabilitation rollator can be dangerous for both the patient and those around them.
- In people with very limited muscle strength, the use of a rehabilitation rollator may be difficult or ineffective. In such cases, a different approach to rehabilitation may be necessary, such as manual therapy or Muscle Strength Support Exercises.
- In patients with severe balance disorders, the use of a rollator can be dangerous because it can increase the risk of falls and further injury.

USE

INSTALLATION OF A WALKER

Step 1: Mount the front wheels, using the nut and allen wrench.

Note: DO NOT use the rollator unless the wheels are securely locked.

Step 2: Lift the frame of the rollator with the handles, then pull the red latches to properly lock the frame. You will hear a characteristic click.

Step 3: Open the rehabilitation rollator by pulling the handles away from each other .

Step 4: Press the seat , making sure the seat arms snap into the seat supports.

Note: DO NOT use the rollator if the seat arms are not fully locked into the seat supports.

Note: DO NOT use the product if ANY components of the rollator have not been securely locked.

Step 5: Install the bag holders.

BACKREST ASSEMBLY

Slide the backrest into the space provided.

HEIGHT ADJUSTMENT

The handles can be raised or lowered to customize their height.

To adjust the height of the handles, press the height adjustment button. Raise or lower the handles to the desired user height. You will hear a click when the handle locks into one of the height adjustment positions.

BEFORE USE, MAKE SURE THAT BOTH HANDLES ARE AT THE SAME HEIGHT.

DO NOT USE THE ROLLATOR UNLESS THE HANDLES ARE LOCKED IN ONE OF THE HEIGHT ADJUSTMENT POSITIONS.

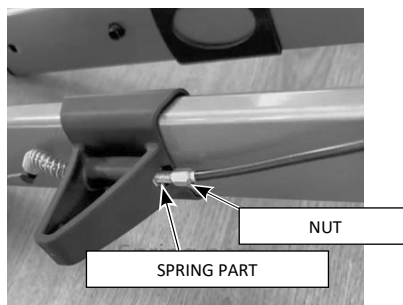
USE OF BRAKE

1. To stop, gently press both brakes simultaneously, which will slow down and stop the rollator.
2. The brakes of the rollator have a parking option. To activate the parking option, press both brakes down simultaneously. The brake will give a very light click. The brakes are now locked.
3. To unlock, just pull the brake up again, you will hear a slight click when "parking" is turned off.

BRAKE ADJUSTMENT

The brake system of the rollator will require periodic adjustment. To loosen the brakes, adjust the nut .
To tighten the brakes, adjust the nut to lengthen the spring part.
If you need to loosen the brakes, adjust the nut so that the spring part is shorter.

**FEATURES**

- aluminum frame
- aluminum bracket
- 4 pieces of 12-inch pneumatic wheels with rubber tires,
- Nylon seat and removable backrest,
- aluminum front fork,
- Nylon shopping bag with safety reflector,
- Detachable cane holder and water bottle holder
- handbrake,
- Quick folding and unfolding function, convenient to fold and store,

SPECIFICATIONS

Seat size: **45 x 23 cm**

Seat height: **60 cm**

Seat depth: **23cm**

Seat width: **45 cm**

Size of the unfolded rollator:

Width: **68 cm**

Length: **80.5 cm**

Adjustable handle height: **77-96 cm (in 9 positions)**

Wheel size: **29x4 cm**

Weight: **10,6 kg**

Maximum load: **136 kg**

shopping bag load: **5kg**

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER!

COMPONENTS: Rehabilitation rollator, instruction manual, Organizer, bag mounting set, first aid kit (keychain), bag, bell, flashlight, bag holders, water holder, cane holders, allen wrench **NOTES:**

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

NOTES: If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTE: It is forbidden to use the product other than for its intended purpose .

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

Use the rollator only as intended, that is, as a walking aid. It is intended for use only on standard sidewalks or at home. Do not use it on uneven terrain or surfaces with holes. Using the rollator on such surfaces can seriously jeopardize your safety and cause irreparable damage to the rollator. If any of the wheels become difficult to turn or are not securely attached to the frame, or if you suspect damage or any other malfunction, do not use the rehabilitation rollator.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING & CARE

Clean the rollator regularly with lukewarm soapy water. Do not use strong detergents, do not use rough objects for cleaning. Store in a dry and cool room away from direct sunlight.

Keep the rollator clean by wiping it with a damp cloth and mild detergent. Under no circumstances use cleaners containing abrasives or solvents, as this can cause serious damage to the rollator.

Wipe all surfaces with a dry cloth and use a water dispersant, such as WD40, on all metal joints.

Regularly check that all wheels and the folding mechanism work properly and that all wheels move freely. Pay special attention to the front wheels, making sure they turn freely and without obstruction and that they are securely attached to the frame.

Monthly inspections - check that no component is loose. Check brake adjustment. Check the wheels.

Note: Do not install any parts or accessories to your rollator manufactured by another company. Doing so will void the warranty and could seriously jeopardize your safety.

We suggest a daily visual inspection of the rollator to check that all screws are properly tightened and that the rollator and its components have no damage.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

METHOD OF DISPOSAL OF THE PRODUCT

Once the product is decommissioned, the medical device can be disposed of as normal municipal waste. To dispose of it properly, please contact your local authority or the nearest waste collection point to find out how to dispose of the product properly. By acting in accordance with local regulations, you are helping to protect the environment.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o. , 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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