AT51090
INSTRUKCJA UŻYTKOWNIA, CHODZIK
USER MANUAL ROLLATOR
BEDIENUNGSANLEITUNG ROLLATOR
UŽIVATELSKÁ PŘÍRUČKA ROLLATOR
NÁVOD NA POUŽITIE ROLÁTOR
MANUEL D'INSTRUCTION DU DÉAMBULATEUR
HANDLEIDING ROLLATOR
MANUAL DE USUARIO ROLLATOR
MANUALE D'USO DEL ROLLATOR
ANVÄNDARMANUAL ROLLATOR

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EN

Thank you for purchasing our product.

Before starting use, please carefully read this instruction manual. Improper use may result in personal injury, damage, or accidents. Therefore, to ensure safe and proper use of the product, please familiarize yourself thoroughly with this manual.

WARNING: All parts must be checked for damage that may have occurred during transport. If any such damage is noticed, the product **MUST NOT** be used. For further information, please contact the manufacturer.

TARGET PATIENT GROUP

People suffering from diseases, dysfunctions, or injuries for which this product is intended for treatment/rehabilitation or compensation (see the section on the intended purpose of the product in this manual). The product may be purchased independently by the user or based on the recommendation of a physician, therapist, or other specialist. Whether purchased independently or recommended by a healthcare professional, available sizes, necessary functions, product variants, indications and contraindications for use, as well as information provided by the manufacturer should be taken into account.

SAFETY RECOMMENDATIONS

To ensure safety during the use of the walker, please follow the guidelines below.

GENERAL WARNING

Do not assemble or use the product before reading and understanding this manual.

If you do not understand the warnings, rules, or instructions, contact a healthcare professional, distributor, or technical manufacturer before attempting to assemble the product to avoid injury or product damage.

Each user should consult their physician or therapist to determine the appropriate size, adjustments, and method of use. The walker is not intended for sitting on it without supervision or for use at unstable heights.

The walker should be evaluated by a physician or therapist to determine appropriate user support, correct brake settings, and required operating force. If the walker is intended to carry a bag, make sure the safety bar is locked and the bag is properly placed inside the basket. Ensure that the handles are correctly adjusted. After folding, the walker should remain stable.

The quick-release mechanism is installed in the correct position and remains locked under normal conditions of use. The metal pins in the adjustment tube extend outward; applying force to these components may cause injury. Before use, ensure that all wheel adjustment components and moving parts are in good condition. During use, all wheels must remain in contact with the ground. Always exercise caution to ensure walker stability. When the walker is used in a locked position, the hand brakes **MUST** be engaged.

WARNING:

During use, handling, folding, and adjustment of the product, there is a risk of trapping and/or pinching body parts of the user or accompanying person in openings or gaps between components. Perform these actions with particular care. After completing adjustments, stabilize the position by securely tightening all nuts/screws.

INTENDED PURPOSE

The wheeled walker is intended for people with reduced mobility who retain sufficient walking ability and balance, enabling them to leave their place of residence with the support of the wheeled walker.

APPLICATION / INDICATIONS

The product is designed to support mobility for individuals with limited walking ability. The rehabilitation rollator is designed for single-person use. It is suitable for:

Balance training: Allows patients to practice maintaining balance, which is crucial in rehabilitation after injuries or neurological disorders.

Posture support: Patients can train correct posture, especially important for individuals with postural defects or stability issues.

Improving motor coordination: Exercises performed with the rollator can help improve coordination and movement control, which is vital for patients with neurological conditions.

Rehabilitation after injuries: Assists in recovery following lower limb injuries, helping patients gradually regain mobility.

Increasing independence: For some patients, the rollator can serve as a tool to increase independence in daily life by enabling self-mobility.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe use of the product.

Using a rehabilitation walker may be unsuitable for individuals with serious spinal stability problems or spinal injuries, as it may increase the risk of further injury. In patients with uncontrolled seizures or epilepsy, use of the rollator may pose risks to both the patient and those nearby.

For individuals with very limited muscle strength, use of the rollator may be difficult or ineffective. In such cases, alternative rehabilitation methods, such as manual therapy or strength-supporting exercises, may be necessary.

For patients with severe balance disorders, using the rollator may increase the risk of falls and additional injuries.

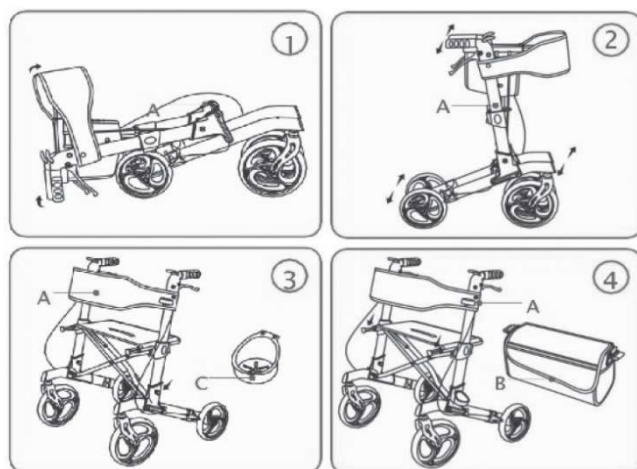
WARNINGS REGARDING ASSEMBLY AND INSTALLATION

After unfolding or assembling the walker, ensure that all components are correctly installed.

PACKAGE CONTENTS: Instruction manual, walker, backpack, bell, first aid pouch, bag, cane holder.

UNFOLDING

Unfold the upper part. Unfold the backrest by pressing the seat downward from above until a click is heard. Install the cane/umbrella holder. Install the bag.

FOLDING

Perform the steps in reverse order to those described in the UNFOLDING section.

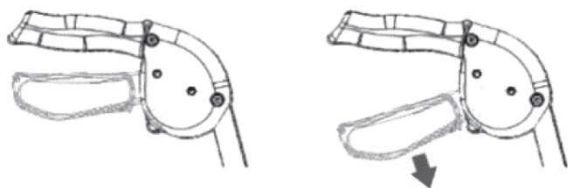
USING THE HAND BRAKES

Pull the brake lever toward the hand grips to slow down.

Perform one of the following actions:

A. STOP – hold the brake lever up.

B. DRIVING – release the brake lever.

LOCKING THE HAND BRAKES

Push the lower part of the brake lever downward as shown until a clicking sound is heard.

UNLOCKING THE HAND BRAKES

Pull the brake lever upward until the latch is released.

Release the brake lever.

Ensure that the brake cable runs along the side tube and is not wrapped around the front of the walker.

TECHNICAL PARAMETERS

Overall width: 64 cm

Overall length: 82 cm

Overall height: 92.5 cm

Seat width: 45 cm

Seat depth: 23.5 cm

Seat height: 55 cm

Rear wheels: 8-inch PVC wheels

Front wheels: 8-inch PVC wheels

Weight: 8.1 kg

Package size: 86 × 26 × 37 cm

Adjustable handle height: 79–91 cm

Maximum user weight: 136 kg

This symbol indicates the maximum user weight.

CLEANING

Clean the walker with a damp cloth or sponge and warm water with a mild detergent for routine cleaning. Ensure that no cleaning residue remains on the handles or seat (if installed), as this may cause damage.

Do not use abrasive sponges, polishing agents, or chemical solvents.

MAINTENANCE

Ensure that all fastening elements are always securely tightened.

Check the screws securing the seat to the side elements and tighten if necessary.

Replace all damaged or worn components immediately.

WHEEL MAINTENANCE

Clean the wheels with warm water and a mild cleaning agent. Dry with a soft cloth.

Periodically lubricate with 3-in-1 oil if the wheels begin to squeak.

IMPORTANT NOTICE

In the event of a serious incident related to the product that directly or indirectly led, could have led, or may lead to:

- a) death of the patient, user, or another person, or
- b) temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) a serious public health threat,

the incident must be reported to the manufacturer and the competent authority of the EU Member State where the user or patient resides.

WARNING: If pain, allergic reactions, or other disturbing or unclear symptoms occur during the use of the medical device, consult a healthcare professional.

WARNING: Use of the product for purposes other than its intended use is prohibited.

TRANSPORT

The rollator folds easily, making it convenient to transport. For transportation, fold it as described above. Always transport the rollator securely to prevent sudden, unsafe movements.

REUSE

The rollator is suitable for reuse. Before passing it to another user, it must be inspected by a qualified dealer. Check the following:

Visual condition of the frame.

Secure fit of screws and nuts.

Condition of plastic parts.

Condition of wheels and bearings.

Function and condition of brakes.

If any part is damaged or broken, it must be replaced. Wheels should be replaced whenever significant wear is noticed. All loosened screws and nuts must be tightened. The product must be thoroughly cleaned and disinfected before reuse.

WARNING: Do not reuse the rollator if the frame is bent or broken.

The user manual must always accompany the rollator when changing users.

STORAGE

Store the product in a dry place. Do not expose it to direct sunlight.

DISPOSAL

When taken out of service, dispose of the product as regular household waste. Follow local waste separation regulations. Protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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